

MAGLUMI® Progesterone (CLIA)

INTENDED USE

The kit is an *in vitro* chemiluminescence immunoassay for the quantitative determination of Progesterone (PROG) in human serum and plasma using the MAGLUMI series Fully-auto chemiluminescence immunoassay analyzer and Biolumi series Integrated System, and the assay is used for an aid in diagnosis and treatment of individuals with suspected or confirmed disorders of the ovaries or placenta.

SUMMARY

Progesterone is an essential hormone in the process of reproduction. It is involved in the menstrual cycle, implantation and is essential for pregnancy maintenance^{1,2}. Luteal phase deficiency (LPD) is a condition of insufficient progesterone exposure to maintain a normal secretory endometrium and allow for normal embryo implantation and growth³, and has been recognized as a specific form of infertility with an incidence usually recorded at about 3.5% in an infertile population⁴. Inadequate secretion of progesterone in early pregnancy has been linked to the etiology of miscarriage and progesterone supplementation has been used as a treatment for threatened miscarriage to prevent spontaneous pregnancy loss⁵. Progesterone levels were 48% lower in women experiencing threatened miscarriage with subsequent spontaneous miscarriage compared to women who delivered at term⁶. Using serum progesterone level in women presenting with threatened miscarriage allowed accurate risk stratification as well as reduction in the use of oral progestogen treatment without significant difference in miscarriage rate⁷. It is suggested that, as long as ultrasonography fails to show an intrauterine pregnancy, the simultaneous determination of serum human chorionic gonadotropin and progesterone could aid in the early diagnosis of ectopic pregnancy and in the improvement of subsequent fertility; to that regard any progesterone level below 15 ng/ml in the presence of detectable amounts of human chorionic gonadotropin is highly suggestive of either a threatened abortion or an ectopic pregnancy, whatever the gestational age⁸.

TEST PRINCIPLE

Competitive chemiluminescence immunoassay. The sample, buffer, magnetic microbeads coated with PROG antigen, ABEI labeled with anti-PROG antibody are mixed thoroughly and incubated. PROG present in the sample compete with PROG antigen immobilized on the magnetic microbeads for binding anti-PROG antibody labeled with ABEI, and form immuno-complexes. After precipitation in a magnetic field, the supernatant is decanted and then perform a wash cycle. Subsequently, the Starter 1+2 are added to initiate a chemiluminescent reaction. The light signal is measured by a photomultiplier as relative light units (RLUs), which is inversely proportional to the concentration of PROG present in the sample.

REAGENTS

Kit Contents

Component	Description	100 tests/kit	50 tests/kit	30 tests/kit
Magnetic Microbeads	Magnetic microbeads coated with PROG antigen (~30.0 µg/mL) in PBS buffer, NaNa ₃ (<0.1%).	2.5 mL	1.5 mL	1.0 mL
Calibrator Low	A low concentration of PROG antigen in PBS buffer, NaNa ₃ (<0.1%).	1.0 mL	1.0 mL	1.0 mL
Calibrator High	A high concentration of PROG antigen in PBS buffer, NaNa ₃ (<0.1%).	1.0 mL	1.0 mL	1.0 mL
Buffer	PBS buffer, NaNa ₃ (<0.1%).	9.5 mL	5.5 mL	3.9 mL
ABEI Label	ABEI labeled with anti-PROG antibody (~0.278 µg/mL) in PBS buffer, NaNa ₃ (<0.1%).	6.5 mL	4.0 mL	3.0 mL
Control 1	A low concentration of PROG antigen (1.00 ng/mL) in PBS buffer, NaNa ₃ (<0.1%).	1.0 mL	1.0 mL	1.0 mL
Control 2	A high concentration of PROG antigen (20.0 ng/mL) in PBS buffer, NaNa ₃ (<0.1%).	1.0 mL	1.0 mL	1.0 mL

All reagents are provided ready-to-use.

Warnings and Precautions

- For *in vitro* diagnostic use.
 - For professional use only.
 - Exercise the normal precautions required for handling all laboratory reagents.
 - Personal protective measures should be taken to prevent any part of the human body from contacting samples, reagents, and controls, and should comply with local operating requirements for the assay.
 - A skillful technique and strict adherence to the package insert are necessary to obtain reliable results.
 - Do not use kit beyond the expiration date indicated on the label.
 - Do not interchange reagent components from different reagents or lots.
 - Avoid foam formation in all reagents and sample types (specimens, calibrators and controls).
 - All waste associated with biological samples, biological reagents and disposable materials used for the assay should be considered potentially infectious and should be disposed of in accordance with local guidelines.
 - This product contains sodium azide. Sodium azide may react with lead or copper plumbing to form highly explosive metal azides. Immediately after disposal, flush with a large volume of water to prevent azide build-up. For additional information, see Safety Data Sheets available for professional user on request.
- Note: If any serious incident has occurred in relation to the device, please report to Shenzhen New Industries Biomedical Engineering Co., Ltd. (Snibe) or our authorized representative and the competent authority of the Member State in which you are established.

Reagent Handling

- To avoid contamination, wear clean gloves when operating with a reagent kit and sample. When handling reagent kit, replace the gloves that have been in contact with samples, since introduction of samples will result in unreliable results.
- Do not use kit in malfunction conditions; e.g., the kit leaking at the sealing film or elsewhere, obviously turbid or precipitation is found in reagents (except for Magnetic Microbeads) or control value is out of the specified range repeatedly. When kit in malfunction conditions, please contact Snibe or our authorized distributor.
- To avoid evaporation of the liquid in the opened reagent kits in refrigerator, it is recommended that the opened reagent kits to be sealed with reagent seals contained within the packaging. The reagent seals are single use, and if more seals are needed, please contact Snibe or our authorized distributor.
- Over time, residual liquids may dry on the septum surface. These are typically dried salts and have no effect on assay efficacy.
- Use always the same analyzer for an opened reagent integral.
- For magnetic microbeads mixing instructions, refer to the Preparation of the Reagent section of this package insert.
- For further information about the reagent handling during system operation, please refer to Analyzer Operating Instructions.

Storage and Stability

- Do not freeze the integral reagents.
- Store the reagent kit upright to ensure complete availability of the magnetic microbeads.
- Protect from direct sunlight.

Stability of the Reagents	
Unopened at 2-8°C	until the stated expiration date
Opened at 2-8°C	6 weeks
On-board	4 weeks

Stability of Controls	
Unopened at 2-8°C	until the stated expiration date
Opened at 10-30°C	6 hours
Opened at 2-8°C	6 weeks
Frozen at -20°C	3 months
Frozen and thawed cycles	no more than 3 times

SPECIMEN COLLECTION AND PREPARATION

Specimen Types

Only the specimens listed below were tested and found acceptable.

Specimen Types	Collection Tubes
Serum	Tubes without additive/accessory, or tubes containing clot activator or clot activator with gel.
Plasma	K2-EDTA

- The sample types listed were tested with a selection of sample collection tubes that were commercially available at the time of testing, i.e. not all available tubes of all manufacturers were tested. Sample collection systems from various manufacturers may contain differing materials which could affect the test results in some cases. Follow tube manufacturers' instructions carefully when using collection tubes.

Specimen Conditions

- Do not use grossly hemolyzed/hyperlipidaemia specimens and specimens with obvious microbial contamination.
- Ensure that complete clot formation in serum specimens has taken place prior to centrifugation. Some serum specimens, especially those from patients receiving anticoagulant or thrombolytic therapy, may exhibit increased clotting time. If the serum specimen is centrifuged before a complete clotting, the presence of fibrin may cause erroneous results.
- Samples must be free of fibrin and other particulate matter.
- To prevent cross contamination, use of disposable pipettes or pipette tips is recommended.

Preparation for Analysis

- Inspect all specimens for foam. Remove foam with an applicator stick before analysis. Use a new applicator stick for each specimen to prevent cross contamination.
- Frozen specimens must be completely thawed before mixing. Mix thawed specimens thoroughly by low speed vortexing or by gently inverting. Visually inspect the specimens. If layering or stratification is observed, mix until specimens are visibly homogeneous. If specimens are not mixed thoroughly, inconsistent results may be obtained.
- Specimens should be free of fibrin, red blood cells, or other particulate matter. Such specimens may give reliable results and must be centrifuged prior to testing. Transfer clarified specimen to a sample cup or secondary tube for testing. For centrifuged specimens with a lipid layer, transfer only the clarified specimen and not the lipemic material.
- The sample volume required for a single determination of this assay is 10 µL.

Specimen Storage

Specimens removed from the separator, red blood cells or clot may be stored up to 24 hours at 10-30°C, or 2 days at 2-8°C, or 6 months frozen at -20°C. Frozen specimens subjected to up to 2 freeze/thaw cycles have been evaluated.

Specimen Shipping

- Package and label specimens in compliance with applicable local regulations covering the transport of clinical specimens and infectious substances.
- Do not exceed the storage limitations listed above.

Specimen Dilution

- Samples, PROG concentrations above the analytical measuring interval, can be diluted with manual dilution procedure. The recommended dilution ratio is 1:10. The concentration of the diluted sample must be >8.00 ng/mL.
- For manual dilution, multiply the result by the dilution factor.
- Please choose applicable diluents or ask Snibe for advice before manual dilution.

PROCEDURE

Materials Provided

Progesterone (CLIA) assay, control barcode labels.

Materials Required (But Not Provided)

- General laboratory equipment.
- Fully-auto chemiluminescence immunoassay analyzer Maglumi 600, Maglumi 800, Maglumi 1000, Maglumi 2000, Maglumi 2000 Plus, Maglumi 4000, Maglumi 4000 Plus, MAGLUMI X3, MAGLUMI X6, MAGLUMI X8, or Integrated System Biolumi 8000 and Biolumi CX8.
- Additional accessories of test required for the above analyzers include Reaction Module, Starter 1+2, Wash Concentrate, Light Check, Tip, and Reaction Cup. Specific accessories and accessories' specification for each model refer to corresponding Analyzer Operating Instructions.
- Please use accessories specified by Snibe to ensure the reliability of the test results.

Assay Procedure

Preparation of the Reagent

- Take the reagent kit out of the box and visually inspect the integral vials for leaking at the sealing film or elsewhere. If there is no leakage, please tear off the sealing film carefully.
- Open the reagent area door; hold the reagent handle to get the RFID label close to the RFID reader (for about 2s); the buzzer will beep; one beep sound indicates successful sensing.
- Keeping the reagent straight insert to the bottom along the blank reagent track.
- Observe whether the reagent information is displayed successfully in the software interface, otherwise repeat the above two steps.
- Resuspension of the magnetic microbeads takes place automatically when the kit is loaded successfully, ensuring the magnetic microbeads are totally resuspended homogenous prior to use.

Assay Calibration

- Select the assay to be calibrated and execute calibration operation in reagent area interface. For specific information on ordering calibrations, refer to the calibration section of Analyzer Operating Instructions.
- Execute recalibration according to the calibration interval required in this package insert.

Quality Control

- When new lot used, check or edit the quality control information.
- Scan the control barcode, choose corresponding quality control information and execute testing. For specific information on ordering quality controls, refer to the quality control section of the Analyzer Operating Instructions.

Sample Testing

After successfully loading the sample, select the sample in interface and edit the assay for the sample to be tested and execute testing. For specific information on ordering patient specimens, refer to the sample ordering section of the Analyzer Operating Instructions.

To ensure proper test performance, strictly adhere to Analyzer Operating Instructions.

Calibration

Traceability: This method has been standardized against the USP reference standard (Catalog number: 1568007).

Test of assay specific calibrators allows the detected relative light unit (RLU) values to adjust the master curve.

Recalibration is recommended as follows:

- Whenever a new lot of Reagent or Starter 1+2 is used.

- Every 28 days.
- The analyzer has been serviced.
- Control values lie outside the specified range.

Quality Control

Controls are recommended for the determination of quality control requirements for this assay and should be run in singlicate to monitor the assay performance. Refer to published guidelines for general quality control recommendations, for example Clinical and Laboratory Standards Institute (CLSI) Guideline C24 or other published guidelines⁵.

Quality control is recommended once per day of use, or in accordance with local regulations or accreditation requirements and your laboratory's quality control procedures, quality control could be performed by running the Progesterone assay:

- Whenever the kit is calibrated.
- Whenever a new lot of Starter 1+2 or Wash Concentrate is used.

Controls are only applicable with MAGLUMI and Biolumi system and only used matching with the same top seven LOT numbers of corresponding reagents. For each target value and range refer to the label.

The performance of other controls should be evaluated for compatibility with this assay before they are used. Appropriate value ranges should be established for all quality control materials used.

Control values must lie within the specified range, whenever one of the controls lies outside the specified range, calibration should be repeated and controls retested. If control values lie repeatedly outside the predefined ranges after successful calibration, patient results must not be reported and take the following actions:

- Verify that the materials are not expired.
- Verify that required maintenance was performed.
- Verify that the assay was performed according to the package insert.
- If necessary, contact Snibe or our authorized distributors for assistance.

If the controls in kit are not enough for use, please order Progesterone (CLIA) Controls (REF: 160201254MT) from Snibe or our authorized distributors for more.

RESULTS

Calculation

- The analyzer automatically calculates the PROG concentration in each sample by means of a calibration curve which is generated by a 2-point calibration master curve procedure. The results are expressed in ng/mL. For further information please refer to the Analyzer Operating Instructions.

- Conversion factors:
nmol/L x 0.314 = ng/mL (µg/L)
ng/mL x 3.18 = nmol/L

Interpretation of Results

The expected range for the Progesterone assay was obtained by testing 810 apparently healthy individuals in China, gave the following expected value:

Test subjects	N	Mean (ng/mL)	2.5 th percentile (ng/mL)	95 th percentile (ng/mL)	97.5 th percentile (ng/mL)
Males	145	0.875	0.21	/	2.10
Non-pregnant Females	Mid-follicular Phase	137	0.700	0.29	1.55
	Mid-luteal Phase	142	10.813	5.11	18.78
	Post Menopausal	138	0.439	/	0.81
Pregnancy	First Trimester	125	24.724	4.69	51.31
	Second Trimester	123	31.100	19.24	45.55

Results may differ between laboratories due to variations in population and test method. It is recommended that each laboratory establish its own reference interval.

LIMITATIONS

- Results should be used in conjunction with patient's medical history, clinical examination and other findings.
- If the PROG results are inconsistent with clinical evidence, additional testing is needed to confirm the result.
- Heterophilic antibodies in human serum can react with reagent immunoglobulins, interfering with *in vitro* immunoassays. Patients routinely exposed to animals or animal serum products can be prone to this interference and anomalous values may be observed¹⁰.
- Bacterial contamination of the specimens may affect the test results.

SPECIFIC PERFORMANCE CHARACTERISTICS

Representative performance data are provided in this section. Results obtained in individual laboratories may vary.

Precision

Precision was determined using the assay, samples and controls in a protocol (EP05-A3) of the CLSI (Clinical and Laboratory Standards Institute): duplicates at two independent runs per day for 5 days at three different sites using three lots of reagent kits (n = 180). The following results were obtained:

Sample	Mean (ng/mL) (n=180)	Within-Run		Between-Run		Reproducibility	
		SD (ng/mL)	%CV	SD (ng/mL)	%CV	SD (ng/mL)	%CV
Serum Pool 1	2.443	0.083	3.40	0.047	1.92	0.124	5.08
Serum Pool 2	19.635	0.635	3.23	0.401	2.04	0.952	4.85
Serum Pool 3	49.880	1.631	3.27	0.778	1.56	2.298	4.61
Plasma Pool 1	2.551	0.097	3.80	0.018	0.71	0.152	5.96
Plasma Pool 2	20.600	0.655	3.18	0.293	1.42	0.961	4.67
Plasma Pool 3	49.667	1.421	2.86	0.748	1.51	2.112	4.25
Control 1	0.991	0.035	3.53	0.026	2.62	0.061	6.16
Control 2	19.864	0.643	3.24	0.452	2.28	0.975	4.91

Linear Range

0.200-80.0 ng/mL (defined by the Limit of Quantitation and the maximum of the master curve).

Reportable Interval

0.100-800 ng/mL (defined by the Limit of Detection and the maximum of the master curve×Recommended Dilution Ratio).

Analytical Sensitivity

Limit of Blank (LoB) =0.025 ng/mL.

Limit of Detection (LoD) =0.100 ng/mL.

Limit of Quantitation (LoQ) =0.200 ng/mL.

Analytical Specificity

Interference

Interference was determined using the assay, three samples containing different concentrations of analyte were spiked with potential endogenous and exogenous interferents in a protocol (EP7-A2) of the CLSI. The measurement deviation of the interference substance is within ±10%. The following results were obtained:

Interference	No interference up to	Interference	No interference up to
Bilirubin	50 mg/dL	Rheumatoid factor	1500 IU/mL
Hemoglobin	500 mg/dL	ANA	398 AU/mL
Intralipid	2000 mg/dL	Biotin	0.5 mg/dL
17-α Hydroxyprogesterone	10000 ng/mL	Oestrone	50000 ng/mL
Pregnenolone	1000 ng/mL	21-Hydroxyprogesterone	10000 ng/mL
Eltanolone	1000 ng/mL	Bromocriptine	100 ng/mL

Cortisol	20000 ng/mL	Norethindrone	1000 ng/mL
11-Deoxycortisol	6000 ng/mL	19-Nortestosterone	1000 ng/mL
11-Deoxycorticosterone	600 ng/mL	Allopregnanediol	1000 ng/mL
Androstenediol	4000 ng/mL	Clomiphene Citrate	1000 ng/mL
Androstenedione	100 ng/mL	Desogestrel	1000 ng/mL
17-β Estradiol	100 ng/mL	Dihydroandrosterone	1000 ng/mL
Estrilol	100 ng/mL	Ethinylestradiol	1000 ng/mL
Cortisone	600 ng/mL	Ethinodiol Diacetate	1000 ng/mL
Prednisone	10000 ng/mL	Methylprednisolone	1000 ng/mL
Prednisolone	200 ng/mL	Norethindrone Acetate	1000 ng/mL
Medroxyprogesterone	1000 ng/mL	Methylnorethindrone	1000 ng/mL
Danazol	1000 ng/mL	Normethisterone	1000 ng/mL
Estradiol	50000 ng/mL	Spiroolactone	1000 ng/mL
Dexamethasone	10000 ng/mL	11-Ketotestosterone	50 ng/mL
Mesterolone	100 ng/mL	5α-Dihydrotestosterone	100 ng/mL

Cross-Reactivity

Cross-reactivity was determined using the assay, three samples containing different concentrations of analyte were spiked with potential cross-reactants in a protocol (EP7-A2) of the CLSI. The measurement deviation of the interference substance is within ±10%. The following results were obtained:

Cross-reactant	No interference up to	Cross-reactant	No interference up to
Corticosterone	10000 ng/mL	Dehydroepiandrosterone	40000 ng/mL
Testosterone	1000 ng/mL	Dehydroepiandrosterone sulfate	10000 ng/mL
Aldosterone	1000 ng/mL		

Method Comparison

A comparison of the Progesterone assay with a commercially available immunoassay, gave the following correlations (ng/mL):

Number of samples measured: 226

Passing-Bablok: $y=1.0007x+0.0061$, $r=0.990$.

The clinical specimen concentrations were between 0.23 and 78.78 ng/mL.

REFERENCES

1. Di Renzo G C, Giardina I, Clerici G, et al. Progesterone in normal and pathological pregnancy[J]. Hormone Molecular Biology and Clinical Investigation, 2016, 27(1): 35-48.
2. Di Renzo G C, Giardina I, Clerici G, et al. The role of progesterone in maternal and fetal medicine[J]. Gynecological Endocrinology, 2012, 28(11): 925-932.
3. Practice Committee of the American Society for Reproductive Medicine. The clinical relevance of luteal phase deficiency: a committee opinion[J]. Fertility and sterility, 2012, 98(5): 1112-1117.
4. Rosenberg S M, Luciano A A, Riddick D H. The luteal phase defect: the relative frequency of, and encouraging response to, treatment with vaginal progesterone[J]. Fertility and Sterility, 1980, 34(1): 17-20.
5. Wahabi H A, Fayed A A, Esmaeil S A, et al. Progesterone for treating threatened miscarriage[J]. Cochrane Database of Systematic Reviews, 2018 (8).
6. Johns J, Muttukrishna S, Lygnos M, et al. Maternal serum hormone concentrations for prediction of adverse outcome in threatened miscarriage[J]. Reproductive BioMedicine Online, 2007, 15(4): 413-421.
7. Tan T C, Ku C W, Kwek L K, et al. Novel approach using serum progesterone as a triage to guide management of patients with threatened miscarriage: a prospective cohort study[J]. Scientific Reports, 2020, 10(1): 1-7.
8. Hubinont C J, Thomas C, Schwens J F. Luteal function in ectopic pregnancy[J]. American journal of obstetrics and gynecology, 1987, 156(3): 669-674.
9. CLSI. Statistical Quality Control for Quantitative Measurement Procedures: Principles and Definitions. 4th ed. CLSI guideline C24. Wayne, PA: Clinical and Laboratory Standards Institute; 2016.
10. Boscolo L M, Stuart M C. Heterophilic antibodies: a problem for all immunoassays [J]. Clinical Chemistry, 1988,34(1):27-33.

SYMBOLS EXPLANATIONS

	Consult instructions for use		Manufacturer
	Temperature limit (Store at 2-8 °C)		Use-by date
	Contains sufficient for<-> tests		Keep away from sunlight
	This way up		Authorized representative in the European Community
	<i>In vitro</i> diagnostic medical device		Kit component
	Catalogue number		Batch code
	CE marking		

MAGLUMI® and Biolumi® are trademarks of Snibe. All other product names and trademarks are the property of their respective owners.

Shenzhen New Industries Biomedical Engineering Co., Ltd.
No.23, Jinxiu East Road, Pingshan District, 518122 Shenzhen, P.R. China
Tel: +86-755-21536601 Fax: +86-755-28292740

Shanghai International Holding Corp. GmbH (Europe)
Eiffestrasse 80, 20537 Hamburg, Germany
Tel: +49-40-2513175 Fax: +49-40-255726