



## DIAGNOSTIC AUTOMATION, INC.

23961 Craftsman Road, Suite D/E/F, Calabasas, CA 91302

Tel: (818) 591-3030 Fax: (818) 591-8383

[onestep@rapidtest.com](mailto:onestep@rapidtest.com)

[technicalsupport@rapidtest.com](mailto:technicalsupport@rapidtest.com)

[www.rapidtest.com](http://www.rapidtest.com)

IVD



See external label



2°C-8°C



$\Sigma=96$  tests

REF

Cat # 1494-11

# Beta 2 Glycoprotein 1 IgA

Cat # 1494-11

<b>Test</b>	<b>Beta 2 Glycoprotein 1 IgA ELISA</b>
<b>Method</b>	<b>ELISA: Enzyme Linked Immunosorbent Assay</b>
<b>Principle</b>	<b>ELISA - Indirect; Antigen Coated Plate</b>
<b>Detection Range</b>	<b>6.3-200SAU</b>
<b>Sample</b>	<b>5ul</b>
<b>Specificity</b>	<b>91%</b>
<b>Sensitivity</b>	<b>98%</b>
<b>Total Time</b>	<b>~90min</b>
<b>Shelf Life</b>	<b>12 months</b>

*\* Laboratory results can never be the only base of a medical report. The patient history and further tests have to be taken into account*

## NAME AND INTENDED USE

The DIAGNOSTIC AUTOMATION  $\beta_2$ GP1 IgA Enzyme-linked Immunosorbent Assay (ELISA) is intended for the detection and semiquantitative determination of IgA antibodies to  $\beta_2$ GP1 in human sera or plasma. The results of the assay are to be used as an aid in the diagnosis of certain autoimmune disease thrombotic disorders, anti-phospholipid syndrome, SLE or lupus-like disorders.

## SUMMARY AND EXPLANATION OF THE TEST

Cardiolipin autoantibodies (ACA) are described for various autoimmune diseases. The presence of anti-cardiolipin antibodies in systemic lupus erythematosus (SLE) can be related to the development of thrombocytopenia, in gynaecology they are supposed to cause intrauterine death or recurrent abortion. Furthermore, anti-cardiolipin antibodies have been found in some non-thrombotic neurological disorders like cerebrovascular insufficiency, cerebral ischemia or chorea and in myocardial infarction. (1) Recent studies have shown that a 50kD serum cofactor is required for anticardiolipin antibodies, to bind to cardiolipin which has been coated onto plastic plates. The cofactor has been identified as  $\beta_2$ -glycoprotein 1 also termed apolipoprotein H.  $\beta_2$ GP1 has been known as an in vitro inhibitor of the intrinsic blood coagulation pathway, ADP-dependent aggregation, and prothrombinase activity of activated platelets. (2~7) It has become apparent that anticardiolipin antibody from patients with anti-phospholipid syndrome (APS) recognize a modified  $\beta_2$ GP1 structure and not cardiolipin, native  $\beta_2$ GP1 or an epitope structurally defined by both cardiolipin and  $\beta_2$ GP1. (2~6)

Galli et al. (3) and Viard, et al. (8) reported that anti-cardiolipin antibody derived from SLE and APS were directed to the  $\beta_2$ GP1 molecule coated on polystyrene plates. Koike and Matsuura showed conclusively that  $\beta_2$ GP1 is indeed the antigen to which many anticardiolipin antibody patients are actually binding and furthermore showed that the phospholipid merely serves to link the  $\beta_2$ GP1 to the solid phase. (2~9)

$\beta_2$ GP1 autoantibodies are found in the immunoglobulin classes IgG, IgM and IgA. The determination of IgM antibodies is a valuable indicator in the diagnosis of beginning autoimmune disease, whereas IgG and/or IgA antibodies will be found in progressive stages of manifested autoimmune disorders. IgA antibodies are often associated with IgG antibodies. The determination of IgA antibodies seems to have a greater validity in thrombosis and fetal loss. (10). Indications for determination of anti  $\beta_2$ GP1 antibodies are: SLE, Thrombosis, Thrombocytopenia, Cerebral Ischemia, Chorea, Epilepsy, Recurrent Abortion and Intrauterine Death.

## PRINCIPLE OF THE TEST

Purified  $\beta_2$ GP1 antigens are coated on the surface of microwells. Diluted patient serum or plasma, and calibrators, are added to the wells. The Anti  $\beta_2$ GP1 specific antibodies, if present, bind to the antigens. All unbound materials are washed away. After adding enzyme conjugate, it binds to the antibody-antigen complex. Excess enzyme conjugate is washed off, and TMB Chromogenic substrate is added. The enzyme conjugate catalytic reaction is stopped at a specific time. The intensity of the color generated is proportional to the amount of IgA specific antibodies in the sample. The results are read by a microwell reader, and compared in a parallel manner with calibrators.

## STORAGE AND STABILITY

1. Store the kit at 2 - 8o C.
2. Always keep microwells tightly sealed in pouch with desiccants. It is recommended to use up all wells within 4 weeks after initial opening of the pouch.
3. The reagents are stable until expiration of the kit.
4. Do not expose test reagents to heat, sun, or strong light during storage or usage.

## SPECIMEN COLLECTION AND HANDLING

1. Collect blood specimens and separate the serum.
2. Specimens may be refrigerated at 2 - 8o C for up to seven days or frozen for up to six months. Avoid repetitive freezing and thawing of serum sample.

## MATERIALS PROVIDED

- |  |                 |
|--|-----------------|
| 1. Microwell strips: $\beta_2$ GP1 antigen coated wells.   | 12 x 8 wells    |
| 2. Absorbent solution: Black Cap.  | 50 ml / bottle  |
| 3. Washing concentrate 10x.  | 100 ml / bottle |
| 4. TMB Chromogenic Substrate: Amber bottle.  | 15 ml / bottle  |
| 5. Enzyme conjugate: Red color solution.   | 12 ml / bottle  |
| 6. Calibrator set (1:101 prediluted) : 6.3, 12.5, 25, 50, 100, 200 SAU.                                    | 1.5 ml / vial   |
| 7. Control set (1:101 prediluted) : Negative and Positive controls.<br>Ranges are indicated on each label. | 1.5 ml / vial   |
| 8. Stop solution: 1.5 N acid solution. (HCl / H <sub>2</sub> SO <sub>4</sub> )                             | 12 ml / bottle  |

## WARNINGS AND PRECAUTIONS

1. Potential biohazardous materials:  
The calibrator and controls contain human source components, which have been tested and found nonreactive for Hepatitis B surface antigen as well as HIV antibody with FDA licensed reagents. However, as there is no test method that can offer complete assurance that HIV, Hepatitis B virus, or other infectious agents are absent, these reagents should be handled at the Biosafety Level 2, as recommended in the Centers for Disease Control / National Institutes of Health manual, "Biosafety in Microbiological and Biomedical Laboratories." 1984
2. Do not pipette by mouth. Do not smoke, eat, or drink in the areas in which specimens or kit reagents are handled.
3. The components in this kit are intended for use as an integral unit. The components of different lots should not be mixed.
4. This product contains components preserved with sodium azide. Sodium azide may react with lead and copper plumbing to form explosive metal azide. On disposal, flush with a large volume of water.
5. To prevent injury and chemical burns, avoid contact with skin and eyes or inhalation and ingestion of



## QUALITY CONTROL

1. The negative control and positive control should be run with every batch of samples tested and the concentration must be within the range stated on its label.
2. The O.D. value of calibrator 0 SAU must be lower than 0.150 and the O.D. value of calibrator 200 SAU must be greater than 0.750.

**Additional controls may be prepared from human serum specimens and kept under -20°C.**

## INTERPRETATION OF RESULTS

**Each laboratory is recommended to establish its own normal range based upon its own techniques, controls, equipments and patient population according to their own established procedures. The followings are a suggestive guideline.**

Negative: < 20 SAU

Low positive: 20 ~ 40 SAU

Moderate positive: 40 ~ 70 SAU

High positive: > 70 SAU

A positive result suggests the possibility of certain autoimmune disease thrombotic disorders. A negative result indicates no  $\beta_2$ GP1 IgA antibody or levels below the detection limit of the assay.

## LIMITATIONS OF THE TEST

1. Diagnosis cannot be made on the basis of anti  $\beta_2$ GP1 results alone. These results must be used in conjunction with information from clinical evaluation and other diagnostic procedure.
2. The clinical significance of  $\beta_2$  GP1 antibodies in diseases other than SLE is currently under investigation.
3. When negative anti  $\beta_2$  GP1 titers are found in the presence of clinical indications, a lupus anticoagulant, anti-cardiolipin or other additional testing is indicated.
4. It is to be expected that some samples can be anti-cardiolipin positive yet anti  $\beta_2$  GP1 negative. The anti  $\beta_2$  GP1 test is a more specific marker of thrombotic risk. The anticardiolipin test can produce false positive results due to cross-reactivity with dsDNA or certain infectious disease antibodies.

## PERFORMANCE CHARACTERISTICS

### **Sensitivity, specificity, and accuracy:**

A total of 75 samples were assayed with the DIAGNOSTIC AUTOMATION ELISA  $\beta_2$ GP1 IgA (X values) and with a reference ELISA (1) (Y values). The correlation equation is

$$Y = 0.9221 X + 0.5522 \quad R^2 = 0.9375 \quad (n = 75)$$

		Reference ELISA (1)		Total
		N	P	
<b>DIAGNOSTIC AUTOMATION ELISA</b>	N	32 (D)	1 (B)	33
	P	3 (C)	39 (A)	42
<b>β<sub>2</sub>GP1 IgA</b>	Total	35	40	75

Sensitivity =  $A / (A+B) = 39 / (39 + 1) = 98 \%$

Specificity =  $D / (C+D) = 32 / (3 + 32) = 91 \%$

Accuracy =  $(A+D) / (A+B+C+D) = (39 + 32) / (39 + 1 + 3 + 32) = 71 / 75 = 95 \%$

A second reference ELISA (2) was used to test 3 samples which reference ELISA (1) tested for negative and DIAGNOSTIC AUTOMATION ELISA tested for positive. The results are all positive for the 3 samples. The samples tested for positive with reference ELISA (1) and negative with DIAGNOSTIC AUTOMATION ELISA remains a positive result when it was tested with the reference ELISA (2).

### Precision:

Statistic for CV, mean and SD were calculated for each of three samples from the results of 8 determinations in a single run for intra –assay. Inter assay precision was calculated from the result of 8 determinations of 8 different runs.

Intra-assay	n	Mean SAU	SD	% CV
Serum A	8	14.9	0.35	2.38
Serum B	8	30.8	1.39	4.52
Serum C	8	58.9	0.99	1.68

Inter-assay	n	Mean SAU	SD	% CV
Serum A	8	15.6	0.38	2.4
Serum B	8	31.2	1.42	4.55
Serum C	8	59.3	1.05	1.77

## INTERFERENCE AND CROSS-REACTIVITY

DIAGNOSTIC AUTOMATION β<sub>2</sub>GP1 IgA test does not cross-react with the following positive samples tested: Rubella, Toxo, CMV, H. pylori, Measles, Mumps, VZV, RF and HSV.

## REFERENCES

1. Hughes, G.R.V., Harris, E.N. and Gharavi, A.E. The Anticardiolipin Syndrome. The Journal of Rheumatology 1986; 13, 3: 486 – 489.
2. McNeil, H.P., R.J. Simpson, C.N. Chesteman, and S.A. Krilis. Anti-phospholipid antibodies are directed against a complex antigen that includes a lipid-binding inhibitor of coagulation: β<sub>2</sub>-glycoprotein (apolipoprotein H), Proc Natl Acad Sci USA, 87: 4120 – 4124, 1990.

3. Galli, M., P. Comfurius, C. Maassen, H.C. Hemker, M.H. De Baets, P.J.C. Van Breda-Vriesman, T. Barbui, R.F.A. Zwaal, and E.M. Bevers. Anticardiolipin antibodies (ACA) directed not to cardiolipin but to a plasma protein cofactor. *Lancet*, 335: 1544 – 1547, 1990.
4. Matsuura, E., Y. Igarashi, M. Fujimoto, K. Ichikawa, and T. Koike. Anticardiolipin cofactor (s) and differential diagnosis of autoimmune disease. *Lancet*, 336: 177 – 178, 1990.
5. Koike, T. and E. Matsuura. What is the “true” antigen for anticardiolipin antibodies? *Lancet*, 337: 671 – 672, 1991.
6. Matsuura, E., Y. Igarashi, M. Fujimoto, K. Ichikawa, T. Suzuki, T. Sumida, T. Yasuda, and T. Koike. Heterogeneity of anticardiolipin antibodies defined by the anticardiolipin cofactor. *J Immunol*, 148: 3885 – 3891, 1992.
7. Nymph, J., H. Wurm, and G.M. Kostner.  $\beta$ 2-glycoprotein (apo-H) inhibits the release reaction of human platelets during ADP-induced aggregation. *Atherosclerosis*, 63: 109 – 114, 1987.
8. Viard, J.P., Z. Amoura, and J.F. Bach. Association of anti- $\beta$ 2-glycoprotein I antibodies with lupus-type circulating anticoagulant and thrombosis in systemic lupus erythematosus. *Am J Med*, 93: 181 – 186, 1992.
9. Tsutsumi A. et.al. Antibodies to  $\beta$ 2-glycoprotein I and clinical manifestations in patients with Systemic Lupus Erythematosus. *Arthritis and Rheumatism*, 39:1466 –1474, 1996.
10. Lee, B., Siliver, R.M. et. al. Immunoglobulin A anti-  $\beta$ 2GPI antibodies in women who experience unexplained recurrent spontaneous abortion and unexplained fetal death. *Am J Obstet Gynecol*, 9: 748, 2001.

Date Adopted	Reference No.
2005-11-08	DA-Beta 2 Glycoprotein 1 IgA-2009



**DIAGNOSTIC AUTOMATION, INC.**

23961 Craftsman Road, Suite D/E/F, Calabasas, CA 91302

Tel: (818) 591-3030 Fax: (818) 591-8383

**ISO 13485-2003**



Revision Date: 10/07/2009