ACCENT-300 ALKALINE WASHING SOLUTION

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INTENDED USE

ACCENT-300 ALKALINE WASHING SOLUTION is intended to use for cleaning the reagent and sample probes of the biochemistry analyzer ACCENT-300 / BS-300 during routine analyzer operation.

It is only for *in vitro* diagnostics, for healthcare professional users.

OPERATION PRINCIPLE

ACCENT-300 ALKALINE WASHING SOLUTION is a washing solution placed on board the analyzer. Removal of biological material and reagents residues, especially protein-containing materials and microbial deposits from the reagent and sample probes takes place due to the cleaning properties of the reagent. ACCENT-300 ALKALINE WASHING SOLUTION contains sodium hydroxide and sodium hypochlorite that dissolve and remove organic particles.

CONCENTRATION OF THE ACTIVE INGREDIENTS IN THE REAGENT

sodium hydroxide	10.3 mmol/l
sodium hypochlorite	<1%
carbonate buffer stabilizer	

REAGENT STABILITY

The reagent must be used before the expiry date indicated on the label.

WARNING AND NOTES

- Protect from direct sunlight!
- Do not use the reagent beyond expiry date printed on the label.
- Do not combine the remaining reagent with the reagent from a new package.
- Use personal protective equipment to prevent contact with samples, reagents and controls.
- Avoid discharge to sewage system or to environment.
- Please refer to the MSDS for detailed information concerning safe storage and use of the product.
- The product meets the criteria for classification in accordance with Regulation (EC) No 1272/2008.

Ingredients:

ACCENT-300 ALKALINE WASHING SOLUTION contains sodium hypochlorite.

Danger

- H314 Causes severe skin burns and eye damage.
- H318 Causes serious eye damage.
- P280 Wear protective gloves/protective clothing/eye protection/face protection.

P301+P330+P331 IF SWALLOWED: rinse mouth. Do NOT induce vomiting.

P303+P611+P535 IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water.

ACCENT-300 ALKALINE WASHING SOLUTION

P305 +P351 +P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
P310 -Immediately call a POISON CENTER or doctor.

ASSAY PROCEDURE

ACCENT-300 ALKALINE WASHING SOLUTION is ready to use.

Use according to the analyzer's operation manual.

WASTE MANAGEMENT¹

After use, the reagents should be handled as potentially infectious and disposed of in accordance with local legal requirements

- Reagents residues: 18 01 06*
- Empty packages: 15 01 10*
- Wastewater from the analyzer: 18 01 03*

INCIDENTS²

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer (websites address: incidents@cormay.pl) and the competent authority of the Member State in which the user and/or the patient is established.

Serious incident means any incident that directly or indirectly led, might have led or might lead to any of the following:

- the death of a patient, user or other person,
- the temporary or permanent serious deterioration of a patient's, user's or other person's state of health,
- a serious public health threat.

LITERATURE

- European Commission notice on technical guidance on the classification of waste (2018/C 124/01) of 9 April 2018.
- Regulation (EU) 2017/746 of the European Parliament and of the Council of 5 April 2017 on *in vitro* diagnostic medical devices.

LIST OF CHANGES

Previous version: 03	Current version: 04
<i>Sections updated: INTENDED USE, OPERATION PRINCIPLE, CONCENTRATION OF THE ACTIVE INGREDIENTS IN THE REAGENT, WARNING AND NOTES, WASTE MANAGEMENT</i>	
<i>Sections added: REAGENT STABILITY, INCIDENTS, LITERATURE</i>	

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