

AESKULISA Vasculitis Screen

REF 3323

Instruction manual

Contents

1. Intended Use.....	1
2. Clinical Applications and Principle of the Assay.....	1
3. Kit Contents.....	2
4. Storage and Shelf Life.....	2
5. Precautions of Use.....	3
6. Sample Collection, Handling and Storage.....	3
7. Assay Procedure.....	4
8. Quantitative and Qualitative Interpretation.....	5
9. Technical Data.....	6
10. Performance Data.....	6-7
11. Literature.....	7
A : Pipetting scheme.....	8
B : Test Procedure.....	9

1. Intended Use

AESKULISA Vasculitis-Screen is a solid phase enzyme immunoassay for the combined quantitative and qualitative detection of antibodies against PR3 and MPO in human serum. Each well is coated with native human proteinase 3 (PR3) plus native myeloperoxidase (MPO), both highly purified from human peripheral blood polymorphonuclear cells. Anti-PR3 and anti-MPO antibodies recognize specifically conformational epitopes only accessible on the native target antigen.

The assay is an aid in the differential diagnosis of autoimmune vasculitis.

2. Clinical Application and Principle of the Assay

Antibodies against myeloperoxidase (MPO) and proteinase 3 (PR3) belong to the group of anti-neutrophil cytoplasmic antibodies (ANCA) which are directed against cytoplasmic components of neutrophilic granulocytes and monocytes. Indirect immunofluorescence test on ethanol-fixed neutrophils has been the established method for the detection of ANCAs. It became apparent that some ANCAs create a cytoplasmic fluorescence pattern (thus called cANCA) while others create a perinuclear pattern (the pANCA). As both patterns may cover multiple antigens, immunofluorescence is not suitable for a satisfying differential diagnosis of vasculitis; thus each IFT should be verified with specific ELISA tests.

While proteinase 3 is the main antigen specific for cANCA the main antigen for pANCA has been identified as MPO but other cellular components (lactoferrin, cathepsin G, elastase e.g.) may cause perinuclear staining.

MPO is an enzyme from the primary granules of neutrophils with a molecular weight of approximately 140 kDa. Its highly negative charge may be relevant for the location at positively charged structures such as the nuclear membrane and DNA thus responsible for the perinuclear staining pattern of anti-MPO antibodies in patients' sera in IFT using ethanol-fixed neutrophils.

PR3 is a serine protease from the azurophilic granules (lysosomes) of neutrophils with a molecular weight of 29 kDa. The cationic protein has a proteolytic activity to elastin, hemoglobin and collagen VII. In addition, PR3 promotes the activation of platelets by cathepsin G and inactivates the C1-inhibitor.

ANCA are important markers for the differential diagnosis of autoimmune vasculitis. Anti-PR3 are a specific serological marker for Wegener's granulomatosis (WG) and may play an active role in the pathogenesis of WG. Anti-PR3 titers are strongly associated with disease activity and inhibit the proteolytic activity of PR3. Anti-MPO are correlated with idiopathic or vasculitis associated necrotizing crescentic glomerulonephritis and are found frequently in 70% of patients with microscopic polyangiitis, and 5-50% of patients with Churg-Strauss syndrome.

Principle of the test

Serum samples diluted 1:101 are incubated in the microplates coated with the specific antigen. Patient's antibodies, if present in the specimen, bind to the antigen. The unbound fraction is washed off in the following step. Afterwards anti-human immunoglobulins conjugated to horseradish peroxidase (conjugate) are incubated and react with the antigen-antibody complex of the samples in the microplates. Unbound conjugate is washed off in the following step. Addition of TMB-substrate generates an enzymatic colorimetric (blue) reaction, which is stopped by diluted acid (color changes to yellow). The rate of color formation from the chromogen is a function of the amount of conjugate bound to the antigen-antibody complex and this is proportional to the initial concentration of the respective antibodies in the patient sample.

3. Kit Contents

To be reconstituted:

5x Sample Buffer 1 vial, 20 ml - 5x concentrated (capped white: yellow solution)
Containing: Tris, NaCl, BSA, sodium azide < 0.1% (preservative)

50x Wash Buffer 1 vial, 20 ml - 50x concentrated (capped white: green solution)
Containing: Tris, NaCl, Tween 20, sodium azide < 0.1% (preservative)

Ready to use:

Negative Control 1 vial, 1.5 ml (capped green: colorless solution)
Containing: Human serum (diluted), sodium azide < 0.1% (preservative)

Positive Control 1 vial, 1.5 ml (capped red: yellow solution)
Containing: Human serum (diluted), sodium azide < 0.1% (preservative)

Cut-off Calibrator 1 vial, 1.5 ml (capped blue: yellow solution)
Containing: Human serum (diluted), sodium azide < 0.1% (preservative)

Calibrators 6 vials, 1.5 ml each 0, 3, 10, 30, 100, 300 U/ml
(color increasing with concentration: yellow solutions)
Containing: Human serum (diluted), sodium azide < 0.1% (preservative)

Conjugate 1 vial, 15 ml IgG (capped blue: blue solution)
Containing: Anti-human immunoglobulins conjugated to horseradish peroxidase

TMB Substrate 1 vial, 15 ml (capped black)
Containing: Stabilized TMB/H₂O₂

Stop Solution 1 vial, 15 ml (capped white: colorless solution)
Containing: 1M Hydrochloric Acid

Microtiterplate 12x8 well strips with breakaway microwells
Coating see paragraph 1

Material required but not provided:

Microtiter plate reader 450 nm reading filter and optional 620 nm reference filter (600-690 nm). Glass ware(cylinder 100-1000ml), test tubes for dilutions. Vortex mixer, precision pipettes (10, 100, 200, 500, 1000 µl) or adjustable multipipette (100-1000ml). Microplate washing device (300 µl repeating or multi-channel pipette or automated system), adsorbent paper.

Our tests are designed to be used with purified water according to the definition of the United States Pharmacopeia (USP 26 - NF 21) and the European Pharmacopeia (Eur.Ph. 4th ed.).

4. Storage and Shelf Life

Store all reagents and the microplate at 2-8°C/35-46°F, in their original containers. Once prepared, reconstituted solutions are stable for 1 month at 4°C/39°F, at least. **Reagents and the microplate shall be used within the expiry date indicated on each component, only. Avoid intense exposure of TMB solution to light. Store microplates in designated foil, including the desiccant, and seal tightly.**

5. Precautions of Use

5.1 Health hazard data

THIS PRODUCT IS FOR IN VITRO DIAGNOSTIC USE ONLY. Thus, only staff trained and specially advised in methods of in vitro diagnostics may perform the kit. Although this product is not considered particularly toxic or dangerous in conditions of normal use, refer to the following for maximum safety :

Recommendations and precautions

This kit contains potentially hazardous components. Though kit reagents are not classified being irritant to eyes and skin we recommend to avoid contact with eyes and skin and wear disposable gloves.

WARNING ! Calibrators, Controls and Buffers contain sodium azide (NaN_3) as a preservative. NaN_3 may be toxic if ingested or adsorbed by skin or eyes. NaN_3 may react with lead and copper plumbing to form highly explosive metal azides. On disposal, flush with a large volume of water to prevent azide build-up. Please refer to decontamination procedures as outlined by CDC or other local/national guidelines.

Do not smoke, eat or drink when manipulating the kit.

Do not pipette by mouth.

All human source material used for some reagents of this kit (controls, standards e.g.) has been tested by approved methods and found negative for HbsAg, Hepatitis C and HIV 1. However, no test can guarantee the absence of viral agents in such material completely. Thus handle kit controls, standards and patient samples as if capable of transmitting infectious diseases and according to national requirements.

5.2 General directions for use

Do not mix or substitute reagents or microplates from different lot numbers. This may lead to variations in the results.

Allow all components to reach room temperature (20-32°C/68-89.6°F) before use, mix well and follow the recommended incubation scheme for an optimum performance of the test.

Incubation: We recommend test performance at 30°C/86°F for automated systems.

Never expose components to higher temperature than 37°C/ 98.6 °F.

Always pipette substrate solution with brand new tips only. Protect this reagent from light. Never pipette conjugate with tips used with other reagents prior.

A definite clinical diagnosis should not be based on the results of the performed test only, but should be made by the physician after all clinical and laboratory findings have been evaluated. The diagnosis is to be verified using different diagnostic methods.

6. Sample Collection, Handling and Storage

Use preferentially freshly collected serum samples. Blood withdrawal must follow national requirements.

Do not use icteric, lipemic, hemolysed or bacterially contaminated samples. Sera with particles should be cleared by low speed centrifugation (<1000 x g). Blood samples should be collected in clean, dry and empty tubes. After separation, the serum samples should be used immediately, respectively stored tightly closed at 2-8°C/35-46°F up to three days, or frozen at -20°C/-4°F for longer periods.

7. Assay Procedure

7.1 Preparations prior to pipetting

Dilute concentrated reagents:

Dilute the concentrated sample buffer 1:5 with distilled water (e.g. 20 ml plus 80 ml).

Dilute the concentrated wash buffer 1:50 with distilled water (e.g. 20 ml plus 980 ml).

Samples:

Dilute serum samples 1:101 with sample buffer (1x)

e.g. 1000 µl sample buffer (1x) + 10 µl serum. Mix well !

Washing:

Prepare 20 ml of diluted wash buffer (1x) per 8 wells or 200 ml for 96 wells

e.g. 4 ml concentrate plus 196 ml distilled water.

Automated washing:

Consider excess volumes required for setting up the instrument and dead volume of robot pipette.

Manual washing:

Discard liquid from wells by inverting the plate. Knock the microwell frame with wells downside vigorously on clean adsorbent paper. Pipette 300 µl of diluted wash buffer into each well, wait for 20 seconds. Repeat the whole procedure twice again.

Microplates:

Calculate the number of wells required for the test. Remove unused wells from the frame, replace and store in the provided plastic bag, together with desiccant, seal tightly (2-8°C/35-46°F).

7.2 Work flow

For pipetting scheme see Annex A, for the test procedure see Annex B

We recommend pipetting samples and calibrators in duplicate.

Cut-off calibrator should be used for qualitative testing only.

- Pipette 100 µl of each patient's diluted serum into the designated microwells.
- Pipette 100 µl calibrators OR cut-off calibrator and negative and positive controls into the designated wells.
- Incubate for 30 minutes at 20-32°C/68-89.6°F.
- Wash 3x with 300 µl washing buffer (diluted 1:50).
- Pipette 100 µl conjugate into each well.
- Incubate for 30 minutes at 20-32°C/68-89.6°F.
- Wash 3x with 300 µl washing buffer (diluted 1:50).
- Pipette 100 µl TMB substrate into each well.
- Incubate for 30 minutes at 20-32°C/68-89.6°F, protected from intense light.
- Pipette 100 µl stop solution into each well, using the same order as pipetting the substrate.
- Incubate 5 minutes minimum.
- Agitate plate carefully for 5 sec.
- Read absorbance at 450 nm (optionally 450/620 nm) within 30 minutes.

8. Quantitative and Qualitative Interpretation

For **quantitative interpretation** establish the standard curve by plotting the **optical density (OD) of each calibrator (y-axis)** with respect to the corresponding concentration values in **U/ml (x-axis)**. For best results we recommend log/lin coordinates and 4-Parameter Fit. From the OD of each sample, read the corresponding antibody concentrations expressed in **U/ml**.

Normal Range	Equivocal Range	Positive Results
< 12 U/ml	12 - 18 U/ml	> 18 U/ml

Example of a standard curve

We recommend pipetting calibrators in parallel for each run.

Calibrators IgG	OD 450/620 nm	CV % (Variation)
0 U/ml	0.101	4.1
3 U/ml	0.225	1.9
10 U/ml	0.380	4.3
30 U/ml	0.732	0.9
100 U/ml	1.277	0.8
300 U/ml	2.150	0.5

Example of calculation

Patient	Replicate (OD)	Mean (OD)	Result (U/ml)
P 01	1.405/1.383	1.394	110.8
P 02	0.909/0.931	0.920	49.3

For lot specific data, see enclosed quality control leaflet. Medical laboratories might perform an in-house Quality Control by using own controls and/or internal pooled sera, as foreseen by EU regulations.

Do not use this example for interpreting patients results!

Each laboratory should establish its own normal range based upon its own techniques, controls, equipment and patient population according to their own established procedures.

For qualitative interpretation read the optical density of the cut-off calibrator and the patient samples. Compare patient's OD with the OD of the cut-off calibrator. For qualitative interpretation we recommend to consider sera within a range of 20% around the cut-off value as equivocal. All samples with higher ODs are considered positive, samples with lower ODs are considered negative.

Negative: $OD_{\text{patient}} < 0.8 \times OD_{\text{cut-off}}$

Equivocal: $0.8 \times OD_{\text{cut-off}} \leq OD_{\text{patient}} \leq 1.2 \times OD_{\text{cut-off}}$

Positive $OD_{\text{patient}} > 1.2 \times OD_{\text{cut-off}}$

9. Technical Data

Sample material:	serum
Sample volume:	10 µl of sample diluted 1:101 with 1x sample buffer
Total incubation time:	90 minutes at 20-32°C/68-89.6°F
Calibration range:	0-300 U/ml
Analytical sensitivity:	1.0 U/ml
Storage:	at 2-8°C/35-46°F use original vials, only
Number of determinations:	96 tests

10. Performance Data

10.1 Analytical sensitivity

Testing sample buffer 30 times on *AESKULISA Vasculitis Screen* gave an analytical sensitivity of 1.0 U/ml.

10.2 Specificity and Sensitivity

The microplates are coated with ***native human proteinase 3 and native human myeloperoxidase*** highly purified from human neutrophil granulocytes. No crossreactivities to other autoantigens have been found. The diagnostic specificity of PR3 antibodies for Wegener's Granulomatosis is >95%. The diagnostic sensitivity of PR3 antibodies for active Wegener's Granulomatosis is 90% and 75% in the absence of renal involvement.

Anti-MPO are found frequently in 70% of patients with microscopic polyangiitis, and 5-50% of patients with Churg-Strauss syndrome. The data have been acquired with the *AESKULISA Vasculitis Screen (REF7323)*.

Correlation:

The comparability of performance data was assessed with at least 30 sera tested on both, *AESKULISA 7323* and *AESKULISA 3323*. A linear regression analysis of the two products showed that the two products are equivalent. Data can be received upon request.

10.3 Linearity

Chosen sera have been tested with this kit and found to dilute linearly. However, due to the heterogeneous nature of human autoantibodies there might be samples that do not follow this rule.

Sample No.	Dilution Factor	measured concentration (U/ml)	expected concentration (U/ml)	Recovery (%)
1	1 / 100	218.0	220.0	99.1
	1 / 200	105.0	110.0	95.5
	1 / 400	51.4	55.0	93.5
	1 / 800	25.3	27.5	92.0
2	1 / 100	112.6	115.0	97.9
	1 / 200	56.3	57.5	97.9
	1 / 400	27.1	28.8	94.1
	1 / 800	13.4	14.4	93.1

10.4 Precision

To determine the precision of the assay, the variability (intra and inter-assay) was assessed by examining its reproducibility on three serum samples selected to represent a range over the standard curve.

Intra-Assay		
Sample No.	Mean (U/ml)	CV (%)
1	211.4	3.2
2	118.9	2.9
3	88.9	1.5

Inter-Assay		
Sample No.	Mean (U/ml)	CV (%)
1	209.4	3.4
2	120.9	2.5
3	94.3	2.8

10.5 Calibration

Due to the lack of international reference calibration this assay is calibrated in arbitrary units (U/ml).

11. Literature

- Falk, RJ Jennette JC (1988).**
Antineutrophil cytoplasmic autoantibodies with specificity for myeloperoxidase in patients with systemic vasculitis and idiopathic necrotizing and crescentic glomerulonephritis.
N Engl J Med 318: 1651-1657.
- Lüdemann J, Utecht B, Gross WL (1990).**
Antineutrophil cytoplasm antibodies in Wegener's granulomatosis recognize an elastinophil enzyme.
J Exp Med 171: 375-362.
- Csernok E, Muller A, Gross WL (1999).**
Immunopathology of ANCA-associated vasculitis.
Intern Med 38: 759-765.
- Dolman KM, Stegman CA, van de Wiel BA, Hack CE, von dem Borne AE, Kallenberg CG, Goldschmeding R (1993).**
Relevance of classic anti-neutrophil cytoplasmic autoantibody (cANCA)-mediated inhibition of proteinase 3-alpha 1-antitrypsin complexation to disease activity in Wegener's granulomatosis.
Clin Exp Immunol 93: 405-410.
- Goldschmeding R, van der Schoot CE, ten Bokkel Huinink D, et al. (1989)**
Wegener's granulomatosis autoantibodies identify a novel di isopropylfluorophosphate-binding protein in the lysosomes of human neutrophils.
J Clin Invest 84: 1577-1587.

ANNEX A: Pipetting scheme

We suggest pipetting calibrators, controls and samples as follows:

For **quantitative interpretation** use calibrators to establish a standard curve.

For **qualitative interpretation** use cut-off calibrator.

	for quantitative interpretation use calibrators to establish a standard curve						for qualitative interpretation use cut-off calibrator					
	1	2	3	4	5	6	7	8	9	10	11	12
A	CalA	CalE	P1				NC	P2				
B	CalA	CalE	P1				NC	P2				
C	CalB	CalF	P2				CC	P3				
D	CalB	CalF	P2				CC	P3				
E	CalC	PC	P3				PC	...				
F	CalC	PC	P3				PC	...				
G	CalD	NC	...				P1	...				
H	CalD	NC	...				P1	...				

CalA: calibrator A, CalB: calibrator B, CalC: calibrator C, CalD: calibrator D, CalE: calibrator E, CalF: calibrator F

PC: positive control

NC: negative control




CC: Cut-off calibrator

P1: patient 1

P2: patient 2

P3: patient 3

Annex B: Test Procedure

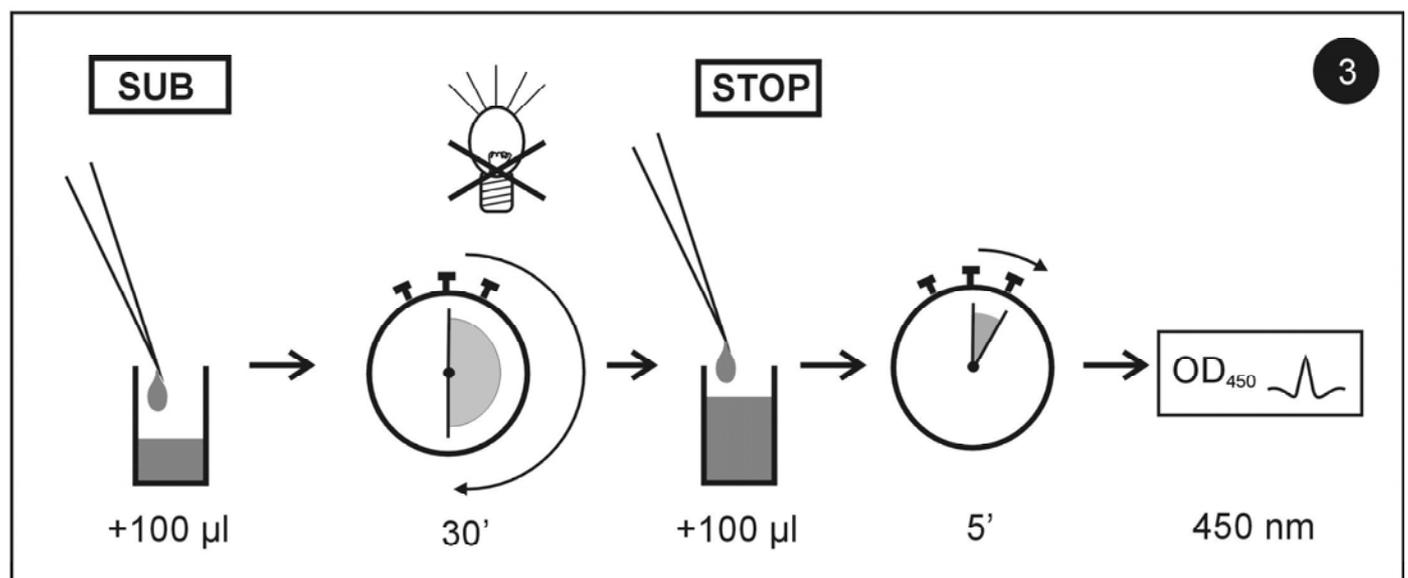
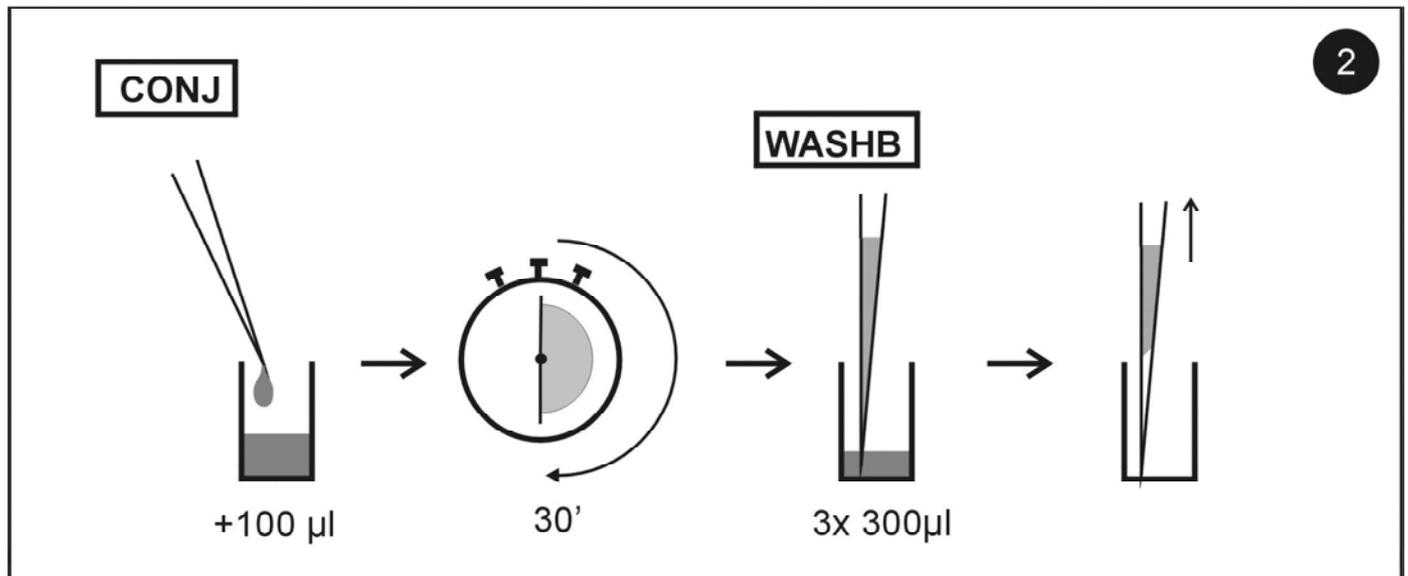
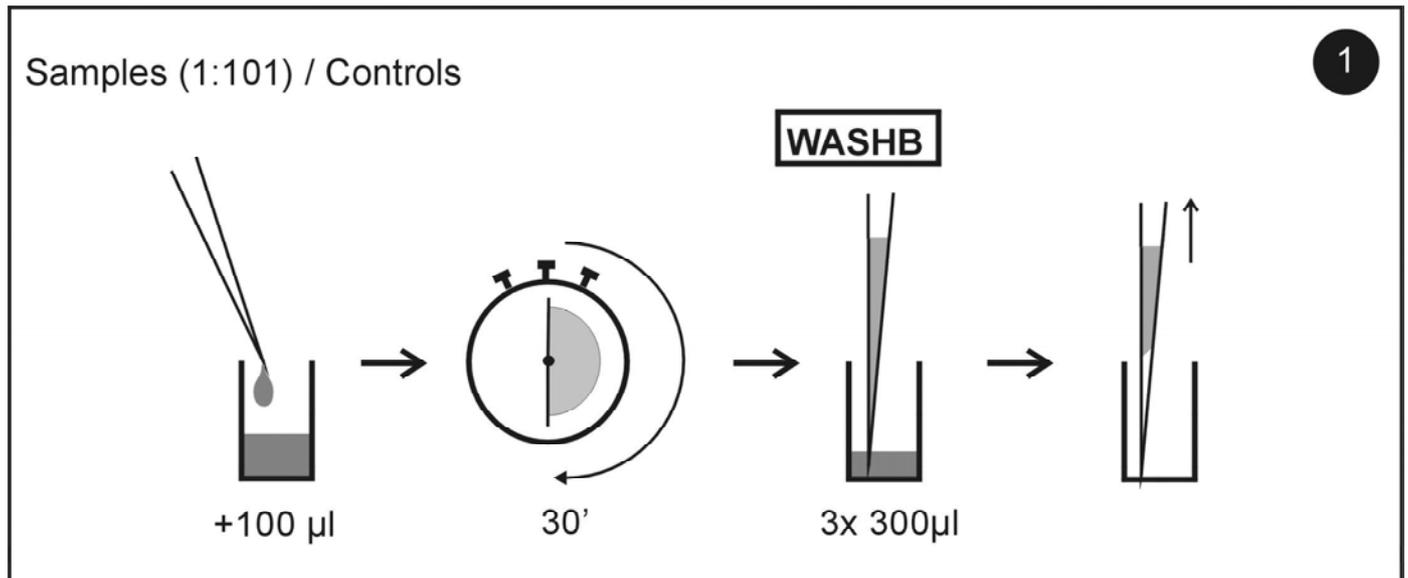


Assay/Test: _____ Incubation / Inkub. : 1. _____ min Date / Datum: _____

Temperature/Temperatur: _____ °F _____ °C 2. _____ min

Name: _____ Signature/Unterschrift: _____ 3. _____ min

	1	2	3	4	5	6	7	8	9	10	11	12
A												
B												
C												
D												
E												
F												
G												
H												

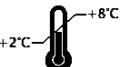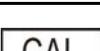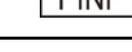	<ul style="list-style-type: none"> ◆ Diagnosi in vitro ◆ Pour diagnostic in vitro ◆ In Vitro Diagnostikum ◆ Para uso Diagnóstico in vitro 	<ul style="list-style-type: none"> ◆ For in vitro diagnostic use ◆ Para uso diagnóstico in vitro ◆ In Vitro Διαγνωστικό μέσο
	<ul style="list-style-type: none"> ◆ Numero d'ordine ◆ Référence Catalogue ◆ Bestellnummer ◆ Número de catálogo 	<ul style="list-style-type: none"> ◆ Catalogue number ◆ Numéro de catálogo ◆ Αριθμός παραγγελίας
	<ul style="list-style-type: none"> ◆ Descrizione lotto ◆ Lot ◆ Chargen Bezeichnung ◆ Lote 	<ul style="list-style-type: none"> ◆ Lot ◆ Lote ◆ Χαρακτηρισμός παρτίδας
	<ul style="list-style-type: none"> ◆ Conformità europea ◆ Déclaration CE de Conformité ◆ Europäische Konformität ◆ Declaração CE de Conformidade 	<ul style="list-style-type: none"> ◆ EC Declaration of Conformity ◆ Declaración CE de Conformidad ◆ Ευρωπαϊκή συμφωνία
	<ul style="list-style-type: none"> ◆ 96 determinazioni ◆ 96 tests ◆ 96 Bestimmungen ◆ 96 Testes 	<ul style="list-style-type: none"> ◆ 96 tests ◆ 96 pruebas ◆ 96 προσδιορισμοί
	<ul style="list-style-type: none"> ◆ Rispettare le istruzioni per l'uso ◆ Voir les instructions d'utilisation ◆ Gebrauchsanweisung beachten ◆ Ver as instruções de uso 	<ul style="list-style-type: none"> ◆ See instructions for use ◆ Ver las instrucciones de uso ◆ Λάβετε υπόψη τις οδηγίες χρήσης
	<ul style="list-style-type: none"> ◆ Da utilizzarsi entro ◆ Utilise avant le ◆ Verwendbar bis ◆ Utilizar antes de 	<ul style="list-style-type: none"> ◆ Use by ◆ Utilizar antes de ◆ Χρήση μέχρι
	<ul style="list-style-type: none"> ◆ Conservare a 2-8°C ◆ Conserver à 2-8°C ◆ Lagerung bei 2-8°C ◆ Conservar entre 2-8°C 	<ul style="list-style-type: none"> ◆ Store at 2-8°C (35-46°F) ◆ Conservar a 2-8°C ◆ Φυλάσσεται στους 2-8°C
	<ul style="list-style-type: none"> ◆ Prodotto da ◆ Fabriqué par ◆ Hergestellt von ◆ Fabricado por 	<ul style="list-style-type: none"> ◆ Manufactured by ◆ Fabricado por ◆ Κατασκευάζεται από
	<ul style="list-style-type: none"> ◆ Calibratore cut-off ◆ Etalon Seuil ◆ Grenzwert Kalibrator ◆ Calibrador de cut-off 	<ul style="list-style-type: none"> ◆ Cut off Calibrator ◆ Calibrador de cut-off ◆ Οριακός ορός Αντιδραστήριο βαθμονόμησης
	<ul style="list-style-type: none"> ◆ Controllo positivo ◆ Contrôle Positif ◆ Positiv Kontrolle ◆ Controlo positivo 	<ul style="list-style-type: none"> ◆ Positive Control ◆ Control Positivo ◆ Θετικός ορός ελέγχου
	<ul style="list-style-type: none"> ◆ Controllo negativo ◆ Contrôle Négatif ◆ Negativ Kontrolle ◆ Controlo negativo 	<ul style="list-style-type: none"> ◆ Negative Control ◆ Control Negativo ◆ Αρνητικός ορός ελέγχου
	<ul style="list-style-type: none"> ◆ Calibratore ◆ Etalon ◆ Kalibrator ◆ Calibrador 	<ul style="list-style-type: none"> ◆ Calibrator ◆ Calibrador ◆ Αντιδραστήριο βαθμονόμησης
	<ul style="list-style-type: none"> ◆ Recupero ◆ Corrélation ◆ Wiederfindung ◆ Recuperação 	<ul style="list-style-type: none"> ◆ Recovery ◆ Recuperado ◆ Ανάκτηση
	<ul style="list-style-type: none"> ◆ Coniugato ◆ Conjugé ◆ Konjugat ◆ Conjugado 	<ul style="list-style-type: none"> ◆ Conjugate ◆ Conjugado ◆ Σύζευγμα
	<ul style="list-style-type: none"> ◆ Micropiastro rivestita ◆ Microplaque sensibilisée ◆ Beschichtete Mikrotiterplatte ◆ Microplaca revestida 	<ul style="list-style-type: none"> ◆ Coated microtiter plate ◆ Microplaca sensibilizada ◆ Επικαλυμμένη μικροπλάκα
	<ul style="list-style-type: none"> ◆ Piastra ad aghi rivestita ◆ Pinplate sensibilisée ◆ Beschichtete Pinplatte ◆ Pinplate revestida 	<ul style="list-style-type: none"> ◆ Coated pinplate ◆ Pinplate sensibilizada ◆ Επικαλυμμένη πλάκα Pin
	<ul style="list-style-type: none"> ◆ Tampone di lavaggio ◆ Tampon de Lavage ◆ Waschpuffer ◆ Solução de lavagem 	<ul style="list-style-type: none"> ◆ Wash buffer ◆ Solución de lavado ◆ Ρυθμιστικό διάλυμα πλύσης
	<ul style="list-style-type: none"> ◆ Tampone substrato ◆ Substrat ◆ Substratpuffer ◆ Substrato 	<ul style="list-style-type: none"> ◆ Substrate buffer ◆ Tampón sustrato ◆ Ρυθμιστικό διάλυμα υποστρώματος
	<ul style="list-style-type: none"> ◆ Reagente bloccante ◆ Solution d'Arrêt ◆ Stopreagenz ◆ Solução de paragem 	<ul style="list-style-type: none"> ◆ Stop solution ◆ Solución de parada ◆ Αντιδραστήριο διακοπής αντίδρασης
	<ul style="list-style-type: none"> ◆ Tampone campione ◆ Tampon Echantillons ◆ Probenpuffer ◆ Diluente de amostra 	<ul style="list-style-type: none"> ◆ Sample buffer ◆ Tampón Muestras ◆ Ρυθμιστικό διάλυμα δειγμάτων