## CHOLESTEROL STANDARD

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### INTENDED USE

CHOLESTEROL STANDARD 200 is dedicated for calibration of cholesterol determination in blood serum and plasma using manual assay.

The standards must be used only for *in vitro* diagnostic, by suitably qualified laboratory personnel, only for the intended purpose, under appropriate laboratory conditions.

### INTRODUCTION

Cholesterol is essential structural component of cell membranes and precursor of bile acids and all steroids hormones. This is why cholesterol has enormous significance for organism normal functioning. But there is also well established association between blood cholesterol concentration and coronary heart disease. Measurement of cholesterol serum level is valuable in prevention and monitoring cardiovascular disease. This determination is useful also for evaluation of intestine absorption, liver and gallbladder function.

### COMPOSITION

CHOLESTEROL STANDARD 200 is single level liquid standard prepared in a buffer base.

The exact standard concentration is printed on the label of each vial.

### PREPARATION AND STORAGE

The standards are ready for use.

Standards when stored at 2-8°C are stable up to expiry date printed on the vial.

### WARNINGS AND NOTES

- Store standards tightly capped when not in use.
- Please refer to the MSDS for detailed information concerning safe storage and use of the product.
- CHOLESTEROL STANDARD 200 meeting the criteria for classification in accordance with Regulation (EC) No 1272/2008

Ingredients:

Standards contain  $\alpha$ -dodecyl- $\omega$ -hydroxy-poly(oxy-1,2-ethanediy).)

### Danger

H318 Causes serious eye damage.

H315 Causes skin irritation.

P280 Wear protective gloves/protective clothing/eye protection/face protection.

P305 + P351 + P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

P310 - Immediately call a POISON CENTER or doctor/physician.

P302 + P352 IF ON SKIN: Wash with plenty of soap and water.

### TRACEABILITY

CHOLESTEROL STANDARD 200 is traceable to the SRM 1951C reference material.

### WASTE MANAGEMENT

Please refer to local legal requirements.

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