ACCENT-300 ALBUMIN

Nr kat. 7-338 (PL)

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Cat. No 7-338 (EN)

INTENDED USE

Diagnostic kit for determination of albumin concentration intended to use in automatic analyser ACCENT-300. The reagents must be used only for *in vitro* diagnostic, by suitably qualified laboratory personnel, only for the intended purpose, under appropriate laboratory conditions.

INTRODUCTION

Albumin is the major serum protein, but is present also in other body fluids: cerebrospinal, pleural and peritoneal. Albumin regulates blood oncotic pressure and serves as amino acids reservoir. Beyond of these functions albumin is very important transport protein – binds and keeps dispersed bilirubin, hormones, vitamins, calcium, magnesium, fatty acids and medicines. Decreased albumin blood level is caused usually by liver or kidney disease, malabsorption or malnutrition.

METHOD PRINCIPLE

Bromocresol green (BCG) forms with albumin, in succinate buffer (acid medium), a coloured complex. The absorbance of this complex is proportional to the albumin concentration in the sample. The colour intensity of the formed complex measured at 630 nm is proportional to albumin concentration in the sample.

REAGENTS

Package

1-Reagent 2 x 40 ml

The reagent is stable up to the kit expiry date printed on the package when stored at 2-8°C. The reagents are stable for 12 weeks on board the analyser at 2-10°C.

Concentrations in the test

succinate buffer ≤ 108 mmol/l
 bromocresol green (BCG) ≤ 0.35 mmol/l
 sodium hydroxide ≤ 60 mmol/l
 detergent
 preservative

Warnings and notes

- Protect from direct sunlight and avoid contamination!
- Do not freeze the reagent.
- Please refer to the MSDS for detailed information concerning safe storage and use of the product.
- 1-Reagent meets the criteria for classification in accordance with Regulation (EC) No 1272/2008.

Warning

H319 Causes serious eye irritation.

P280 Wear protective gloves, eye protection and face protection.

P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

SPECIMEN

Serum free from hemolysis.

Serum should be separated from red blood cells as soon as possible after blood collection.

Serum can be stored up to 3 days at 2-8°C or 6 months at -20°C.²

Nevertheless it is recommended to perform the assay with freshly collected samples!

PROCEDURE

1-Reagent is ready to use.

For reagent blank deionized water is recommended.

Actions required:

When performing assays in analyser ACCENT-300, there is a probability of **cross-contamination** affecting the tests results: ALBUMIN – TG mono, ALBUMIN – UA, ALBUMIN – UA PLUS. To avoid this effect follow the recommendations contained in the instruction 51_03_24_007_ACCENT-300_CARRYOVER.

REFERENCE VALUES⁷

serum		g/dl	g/l
children	3 – 5 years	3.9 – 5.0	39 - 50
	6 – 15 years	4.1 – 5.1	41 - 51
females	16 – 54 years	3.9 – 5.0	39 – 50
males	16 – 29 years	4.6 – 5.3	46 - 53
	30 – 54 years	4.4 – 5.1	44 - 51
females and males	55 – 79 years	4.2 – 5.0	42 – 50

It is recommended for each laboratory to establish its own reference ranges for local population.

QUALITY CONTROL

For internal quality control it is recommended to use, with each batch of samples, the CORMAY SERUM HN (Cat. No 5-172) and CORMAY SERUM HP (Cat. No 5-173). For the calibration of automatic analysers systems the CORMAY MULTICALIBRATOR LEVEL 1 (Cat. No 5-174; 5-176) and LEVEL 2 (Cat. No 5-175; 5-177) are recommended. Deionized water should be used, as a 0 calibrator

The calibration curve should be prepared every 12 weeks, with change of reagent lot number or as required e.g. quality control findings outside the specified range.

PERFORMANCE CHARACTERISTICS

The following results have been obtained using automatic analyser ACCENT-300. Results may vary if a different instrument or a manual procedure is used.

▪ **Sensitivity:** 1.15 g/dl (11.5 g/l).

▪ **Linearity:** up to 6.5 g/dl (65.0 g/l).

For higher concentration, dilute the sample with 0.9% NaCl and repeat the assay. Multiply the result by dilution factor.

Specificity / Interferences

Haemoglobin up to 2.5 g/dl, ascorbate up to 62 mg/l, bilirubin up to 20 mg/dl and triglycerides up to 1200 mg/dl do not interfere with the test.

Precision

Repeatability (run to run) n = 20	Mean [g/dl]	SD [g/dl]	CV [%]
level 1	2.89	0.02	0.82
level 2	4.38	0.03	0.73
Reproducibility (day to day) n = 80	Mean [g/dl]	SD [g/dl]	CV [%]
level 1	2.89	0.03	1.16
level 2	4.41	0.03	0.56

Method comparison

A comparison between albumin values determined at ACCENT-300 (y) and at ADVIA 1650 (x) using 21 samples gave following results:

$y = 1.0159x + 0.0191$ g/dl;

$R = 0.995$ (R – correlation coefficient)

WASTE MANAGEMENT

Please refer to local legal requirements.

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