FLUSH

Nazwa zestawu	(PL) Nr kat.
FLUSH (1L)	8-832
FLUSH (500 ml)	8-891
FLUSH (100 ml)	8-860
FLUSH SET (12 x 17 ml)	8-896

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INTENDED USE

FLUSH is a reagent designed for periodic cleaning the measuring system of haematology and biochemistry analyzers.

The reagent must be used only for *in vitro* diagnostic, by suitably qualified laboratory personnel, only for the intended purpose, under appropriate laboratory conditions.

INTRODUCTION

FLUSH is a strongly alkaline cleaning reagent, based on hypochlorite, used to remove residues of biological material and reagents remaining on the elements of the analyzer measuring system. Reagent was designed for periodic, routine cleaning.

The use of reagent based on sodium hypochlorite is recommended to maintain the analyzer in proper working condition.

Concentrations in the reagent

sodium chloride	≤ 8 g/l
sodium hydroxide	≤ 2.4 g/l
active chlorine	≤ 4.2 %
sodium carbonate	≤ 5.75 g/l

The reagent is stable up to the expiry date printed on the package when stored at 2-25°C.

Warnings and notes

- Protect from direct sunlight and avoid contamination!
- Please refer to the MSDS for detailed information concerning safe storage and use of the product.
- FLUSH/FLUSH SET meets the criteria for classification in accordance with Regulation (EC) No 1272/2008.

Ingredients:

FLUSH/FLUSH SET contains sodium hypochlorite.

Danger

H314 Causes severe skin burns and eye damage.

H318 Causes serious eye damage.

H400 Very toxic to aquatic life.

H412 Harmful to aquatic life with long lasting effects.

P280 Wear protective gloves/protective clothing/eye protection/face protection.

P301+P330+P331 IF SWALLOWED: Rinse mouth. Do NOT induce vomiting.

P303 + P361 + P353 IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water.
P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

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