

Liquid Standards

SODIUM STANDARD SET

2 levels

Standards for the use in tests for the quantitative in vitro determination of sodium in serum on photometric systems

REF	Cont.		
909680	2 x 3 ml	1 x 3 ml L1 1 x 3 ml L2	finished kit
909680SV	2 x 3 ml	1 x 3 ml L1 1 x 3 ml L2	labelled vials (OEM)
909680VSV	2 x 3 ml	1 x 3 ml L1 1 x 3 ml L2	unlabelled vials (OEM)

COMPOSITION

Aqueous buffered solution with reagent grade chemicals and preservatives.

STANDARD PREPARATION

The standards are liquid, ready to use.

STANDARD STABILITY AND STORAGE

Storage: at 2 – 8°C
Stability: until date of expiration date if sealed tightly.

CLOSE IMMEDIATELY AFTER USE!

Do not use the standards if there is visible evidence of microbial growth in the vial.

LOT SPECIFIC ASSAY DATA

LOT: 2203/LNS100109-7		EXP.: 2012/01
Level	Lot	value [mmol/L]
Level 1	2203/LNS100109-7	108.1
Level 2	2203/LNS200109-7	162.2

Values and Expiry Date are LOT specific.

WARNINGS AND PRECAUTIONS

1. This product is for in vitro diagnostic use only.
2. Normal precautions exercised in handling laboratory reagents should be followed.
3. Erroneous results can occur from improper storage, or sample handling errors associated with instrument or assay procedures.

WASTE MANAGEMENT

Please refer to local legal requirements.



DIALAB Produktion und Vertrieb von chemisch – technischen Produkten und Laborinstrumenten Gesellschaft m.b.H.
A – 2351 Wiener Neudorf, Austria
IZ-NÖ Süd, Hondastrasse, Objekt M55
Phone: ++43 (0) 2236 660910-0
Fax: ++43 (0) 2236 660910-30 e-mail: office@dialab.at