Triple Control (ASO, CRP, RF)

Liquid human based control serum for use as a control in immunoturbidimetric assays for ASO, CRP and RF

REF	Cont.	
A04823	1x 1 mL	Triple Control (ASO, CRP, RF)
A04824	1x 5 mL	Triple Control (ASO, CRP, RF)

COMPOSITION

Dialab Triple Control (ASO,CRP,RF) is a dilution of human pleural fluid and serum with high values of ASO, RF and CRP with PBS.
 Sodium azide 0.095 %

PREPARATION

The Triple Control is liquid, ready to use.

STABILITY AND STORAGE

Storage: at 2 – 8 °C
 Stability: up to the expiration date
 Stability after opening: 6 weeks at 2 – 8 °C
 DO NOT FREEZE!

DIRECTIONS FOR USE

1. Use this control in the same manner as specified with the reagent and/or instrument being used.
2. Allow the control to equilibrate to room temperature (22 – 28 °C) prior to use.
3. The control should not be allowed to stand for repeated long periods of time (up to 6 hours) at room temperature. Keep vials tightly capped at all times when not in use to avoid microbial contamination. Keep vials refrigerated when not in use

WARNINGS AND PRECAUTIONS

1. For in Vitro Diagnostic use.
2. Each individual donation intended for use in manufacture of protein calibrator was tested for hepatitis B surface antigen (HBsAg), anti-hepatitis C (anti-HCV) and anti-HIV 1 and HIV 2 by FDA required tests. Since no test method can assure that products derived from human blood do not contain HIV-1/2 and Hepatitis B and Hepatitis C virus, this material and all patient samples should be handled as though capable of transmitting infectious diseases.
3. Reagents containing sodium azide must be handled with due caution: Do not ingest or allow to contact skin or mucous membranes! Sodium azide can form explosive azides when contacting heavy metals such as copper or lead.

WASTE MANAGEMENT

Please refer to local legal requirements.

LOT SPECIFIC ASSAY DATA

Values and expiry date are lot specific.

LOT: TRI017		EXP: 2013/11
Parameter	Target Value	Range
ASO	119 IU/mL (WHO)	101 – 137 IU/mL
	196 IU/mL (SIEMENS)	167 – 225 IU/mL
CRP	3.95 mg/dL (ERM-DA474/IFCC)	3.36 – 4.54 mg/dL
RF	54.6 IU/mL (WHO)	46.4 – 62.8 IU/mL

Protein values have been assigned to ERM-DA474, a Reference Preparation for Protein in Human Serum (RPPHS).
 RPPHS is a certified reference material developed conjointly through the College of American Pathologists (CAP), the Bureau Communautaire de Reference and the International Federation of Clinical Chemists (IFCC).

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