DIACAL LIPIDS

Lyophilized lipid calibration plasma for the use in tests for the quantitative in vitro determination of lipids on photometric systems.

REF

Cont.

D13585SV 1 x 2 ml single vial

COMPOSITION

Diacal Lipids is a lyophilized calibrator based on human plasma with additives of purified material of human origin.

PREPARATION

1. Open the vial very carefully, avoiding any loss of the lyophilized material.
2. Add exactly 2 ml of distilled water (inaccurate reconstitution of the calibrator can cause erroneous results).
3. Close the vial carefully and let it stand for 30 min.
4. Dissolve contents completely by swirling gently, avoiding the formation of foam. Do not shake!

Transfer corresponding quantity needed for calibration into a clean sample cup and treat it the same way as patient samples. Leave frozen aliquots of the reconstituted Diacal Lipids in the dark at room temperature (18 – 25 °C) until they are completely thawed. After thawing, gently swirl for complete homogenization and use directly for calibration.

STABILITY AND STORAGE

Storage:	at 2 – 8 °C		
Stability:	until indicated date of expiration		
Stability after reconstitution:	-20 °C	+4 °C	+ 25 °C
	30 days	5 days	8 hours

Close immediately after use!
 Avoid bacterial contamination!
 Freeze only once!

LOT SPECIFIC CALIBRATOR VALUES

The concentrations of the calibrator analytes are lot-specific and stated in the value sheet of the corresponding lot. Please refer to table with Lot specific assay data. Determinations were performed under standardized protocols using Dialab reagents and Diacal Lipids master calibrator or reference materials. The value sheet contains information about the traceability.

WARNINGS AND PRECAUTIONS

For professional in vitro diagnostic use only. Only blood donations of European origin were used for the production of Diacal Lipids which were found to be non-reactive when tested with approved methods for HBsAg, anti-HIV 1+2 and anti-HCV. Moreover, HCV and HIV were additionally tested by PCR. As there is no possibility to exclude definitely that products derived from human blood components transmit infectious agents, it is recommended to handle the calibrator with the same precautions used for patient specimens.

WASTE MANAGEMENT

Please refer to local legal requirements.

LITERATURE

1. Mayers GL, Cooper GR, Henderson LO, Hassemer DJ, Kimberly MM. Standardization of lipid and lipoprotein measurements. IN: Rifai N, Warnick GR, Dominiczak MH, editors. Handbook of lipoprotein testing. Washington: AACC Press; 1997. p. 223-50.
2. Biosafety in Microbiological and Biomedical Laboratories. U.S. Department of Health and Human Services, Washington 1993 (HHS Publication No. [CDC] 93-8395)

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