Instructions for Use

Zonulin (Stool) ELISA

IVD

REF EIA-5418

 96

DRG 

DRG Instruments GmbH, Germany
Frauenbergstraße. 18, D-35039 Marburg
Phone: +49 (0)6421-1700 0, Fax: +49 (0)6421-1700 50
Website: www.drg-diagnostics.de
E-mail: drg@drg-diagnostics.de

Distributed by:

DRG 

DRG International, Inc., USA
841 Mountain Ave., Springfield, NJ 07081
Phone: (973) 564-7555, Fax: (973) 564-7556
Website: www.drg-international.com
E-mail: corp@drg-international.com

**Please use only the valid version of the Instructions for Use provided with the kit.
Verwenden Sie nur die jeweils gültige, im Testkit enthaltene, Gebrauchsanweisung.
Si prega di usare la versione valida delle istruzioni per l'uso a disposizione con il kit.
Por favor, se usa solo la version valida de la metodico técnico incluido aqui en el kit.**

Table of Contents / Inhaltsverzeichnis / Tabella die Contenuti / Tabla de Contenidos

1	INTENDED USE.....	2
2	SUMMARY AND EXPLANATION OF THE TEST	2
3	MATERIAL SUPPLIED.....	2
4	MATERIAL REQUIRED BUT NOT SUPPLIED	2
5	PREPARATION AND STORAGE OF REAGENTS	3
6	SAMPLE COLLECTION AND PREPARATION.....	4
7	ASSAY PROCEDURE	5
8	RESULTS.....	6
9	LIMITATIONS.....	6
10	QUALITY CONTROL	6
11	PERFORMANCE CHARACTERISTICS.....	7
12	PRECAUTIONS	8
13	TECHNICAL HINTS	8
14	GENERAL NOTES ON THE TEST AND TEST PROCEDURE.....	8
1	VERWENDUNGSZWECK.....	9
2	EINLEITUNG.....	9
3	INHALT DER TESTPACKUNG	9
4	ERFORDERLICHE LABORGERÄTE UND HILFSMITTEL	9
5	VORBEREITUNG UND LAGERUNG DER REAGENZIEN	10
6	PROBENVORBEREITUNG UND -LAGERUNG.....	11
7	TESTDURCHFÜHRUNG	12
8	ERGEBNISSE	13
9	EINSCHRÄNKUNGEN.....	13
10	QUALITÄTSKONTROLLE.....	13
11	TESTCHARAKTERISTIKA.....	14
12	VORSICHTSMASSNAHMEN.....	15
13	TECHNISCHE MERKMALE.....	16
14	ALLGEMEINE HINWEISE ZUM TEST	16
15	REFERENCES / LITERATURE / LITERATUR.....	17
	SYMBOLS USED.....	18

1 INTENDED USE

This ELISA is intended for the quantitative determination of zonulin **in stool**.

For in vitro diagnostic use only.

2 SUMMARY AND EXPLANATION OF THE TEST

Zonulin is a novel human protein analogue to the zonula occludens toxin derived from *Vibrio cholerae* which participates in tight junctions between cells of the wall of the digestive tract. Zonulin binds to a specific receptor on the surface of intestinal epithelia and triggers a cascade of biochemical events which induces tight junction disassembly and a subsequent permeability increase of the intestinal epithelia, allowing some substances to pass through and activate immune reactions.

Fasano and his co-workers found that the zonulin-zonulin-receptor-system is more activated in celiac disease and type 1 diabetes mellitus patients. Patients with active celiac disease showed higher levels of zonulin and anti-zonulin antibodies compared to non-celiac patients and patients in remission, who were on a gluten-free diet.

Concerning the autoimmune type 1 diabetes, in experiments with rats it could be demonstrated that elevated zonulin levels as well as increased intestinal permeability precede a type 1 diabetes disease. Conversely, type 1 diabetes could be prevented by inhibition of zonulin in animal experiments.

In addition, it was reported that many people who suffer from celiac disease also suffer from other autoimmune disorders. It is suggested that increased levels of zonulin are a contributing factor to the development of celiac disease and other autoimmune disorders such as insulin dependent diabetes, multiple sclerosis and rheumatoid arthritis.

3 MATERIAL SUPPLIED

Label	Kit Components	Quantity
PLATE	Microtiter plate, pre-coated	12 x 8 wells
WASHBUF	Wash buffer concentrate, 10x	2 x 100 mL
DIL	Dilution buffer concentrate, 2.5x	2 x 100 mL
TRACER	Tracer concentrate, (biotinylated zonulin)	1 x 300 µL
CONJ	Conjugate concentrate, (peroxidase-labelled streptavidin)	1 x 200 µL
STD	Standards, lyophilized, (see specification for concentrations)	4 x 5 vials
CTRL 1	Control, lyophilized, (see specification for range)	4 x 1 vial
CTRL 2	Control, lyophilized, (see specification for range)	4 x 1 vial
SUB	Substrate (tetramethylbenzidine), ready to use	1 x 15 mL
STOP	Stop solution, ready to use	1 x 15 mL

4 MATERIAL REQUIRED BUT NOT SUPPLIED

- Ultra pure water*
- Stool sample application system such as Cat. No.:EIA-5674
- Calibrated precision pipettors and 10 - 1000 µL single-use tips
- Foil to cover the microtiter plate
- Horizontal microtiter plate shaker
- Multi-channel pipets or repeater pipets
- Vortex mixer
- Standard single-use laboratory glass or plastic vials, cups, etc.
- Microtiter plate reader

*DRG recommends the use of Ultra Pure Water (Water Type 1; ISO 3696), which is free of undissolved and colloidal ions and organic molecules (free of particles > 0.2 µm) with an electrical conductivity < 0.055 µS/cm at 25 °C (≥18.2 MΩ cm).

5 PREPARATION AND STORAGE OF REAGENTS

- To run the assay more than once, ensure that reagents are stored at conditions stated on the label. **Prepare only the appropriate amount necessary for each assay.** The kit can be used up to 4 times within the expiry date stated on the label.
- Reagents with a volume less than **100 µL** should be centrifuged before use to avoid loss of volume.
- **Preparation of the wash buffer:**
The **wash buffer concentrate (WASHBUF)** should be diluted with ultra pure water **1:10** before use (100 mL WASHBUF + 900 mL ultra pure water), mix well.
Crystals could occur due to high salt concentration in the concentrate. Before dilution, the crystals have to be redissolved at room temperature or in a water bath at 37 °C.
The **WASHBUF** is stable at **2 °C - 8 °C** until the expiry date stated on the label.
Wash buffer (1:10 diluted WASHBUF) can be stored in a closed flask at **2 °C - 8 °C for 1 month.**
- **Preparation of the dilution buffer:**
The **dilution buffer concentrate (DIL)** should be diluted with ultra-pure water **1:2.5** before use (100 mL DIL + 150 mL ultra pure water), mix well.
Crystals can occur due to high salt concentration in the concentrate. Before dilution, the crystals must be redissolved in a water bath at 37 °C.
The **DIL** is stable at **2 °C - 8 °C** until the expiry date stated on the label.
Diluted buffer (1:2.5 diluted DIL) can be stored in a closed flask at **2 °C - 8 °C for 1 month.**
- **Preparation of standards and controls:**
The **lyophilized standards (STD)** and **controls (CTRL)** are stable at **2 °C - 8 °C** until the expiry date stated on the label.
Reconstitution details are given in the **specification data sheet.**
Standards and controls (reconstituted STD and CTRL) **are not stable and cannot be stored.**
- **Preparation of the tracer:**
The **tracer concentrate (TRACER)** must be diluted **1:101** in **dilution buffer** (150 µL TRACER + 15 mL dilution buffer) immediately before use.
The **TRACER** is stable at **2 °C - 8 °C** until expiry date stated on the label.
Tracer (1:101 diluted TRACER) **is not stable and cannot be stored.**
- **Preparation of the conjugate:**
The **conjugate concentrate (CONJ)** must be diluted **1:101** in **dilution buffer** (100 µL CONJ + 10 mL dilution buffer) immediately before use.
The **CONJ** is stable at **2 °C - 8 °C** until expiry date given on the label.
Conjugate (1:101 diluted CONJ) **is not stable and cannot be stored.**
- All other test reagents are ready to use. Test reagents are stable until the expiry date (see label) when stored at **2 °C - 8 °C.**

6 SAMPLE COLLECTION AND PREPARATION

6.1 Sample stability

Zonulin is stable in

raw stool for 3 months at -20 °C as well as for 4 days at 2 °C to 8 °C and at room temperature.

stool extracts for 7 days at -20 °C as well as for 4 days at 2 °C to 8 °C.

6.2 Extraction of the stool samples

Dilution buffer is used as a sample extraction buffer. We recommend the following sample preparation:

Stool Sample Application System (SAS) (Cat. No.: EIA-5674)

Stool sample tube – Instruction for use

Please note that the dilution factor of the final stool suspension depends on the used amount of stool sample and the volume of the buffer.

SAS with 0.75 mL dilution buffer:

Applied amount of stool: 15 mg

Buffer Volume: 0.75 mL

Dilution Factor: 1:50

Please follow the instructions for the preparation of stool samples using the SAS as follows:

- The raw stool sample has to be thawed. For particularly heterogeneous samples we recommend a mechanical homogenisation using an applicator, inoculation loop or similar device.
- Fill the **empty stool sample tube** with **0.75 mL of dilution buffer** before using it with the sample.
Important: Allow the dilution buffer to reach room temperature.
- Unscrew the tube (orange part of cap) to open. Insert yellow dipstick into sample. The lower part of the dipstick has notches which need to be covered completely with stool after inserting it into the sample. Place dipstick back into the tube. When putting the stick back into the tube, excess material will be stripped off and leave 15 mg of sample to be diluted. Screw tightly to close the tube.
- Shake the tube well until no stool sample remains in the notches.
Important: Please make sure that you have a maximally homogenous suspension after shaking. Especially with more solid samples, soaking the sample in the tube with dilution buffer for ~ 10 minutes improves the result.
- Allow sample to stand for ~ 10 minutes until sediment has settled. Floating material like shells of grain can be neglected.
- Carefully unscrew the complete cap of the tube including the blue ring plus the dipstick. Discard cap and dipstick. Make sure, that the sediment will not be dispersed again.

Dilution Factor: 1:50

6.3 Sample preparation

Pipette **150 µL of each stool sample supernatant** and **150 µL of tracer** in labelled reaction tubes and mix well.

6.4 Preparation of standards and controls

Transfer **150 µL of STD or CTRL** in the correspondingly labelled reaction tubes, add **150 µL of tracer** and mix well.

7 ASSAY PROCEDURE

7.1 Principle of the test

This assay is based on the method of competitive ELISA.

As a first preparation step, biotinylated zonulin is added to the samples, standards and controls. Afterwards, aliquots of the treated samples, standards and controls are transferred and incubated in microtiter plate wells coated with polyclonal anti-zonulin antibodies. During the incubation, the free target antigen in the samples competes with the biotinylated zonulin for the binding of the polyclonal anti-zonulin antibodies immobilized on the microtiter plate wells. The unbound components are removed by a washing step. During a second incubation step, peroxidase-labeled streptavidin, which binds to the biotinylated zonulin, is added into each microtiter well. After a washing step to remove the unbound components, the peroxidase substrate tetramethylbenzidine is added. Finally, the enzymatic reaction is terminated by an acidic stop solution. The colour changes from blue to yellow and the absorbance is measured in the photometer at 450 nm.

The intensity of the yellow colour is inverse proportional to the zonulin concentration in the sample; this means, high zonulin concentration in the sample reduces the concentration of the biotinylated zonulin bound to the immobilized anti-zonulin antibodies and lowers the photometric signal.

A dose response curve of absorbance unit (optical density, OD at 450 nm) vs. concentration is generated using the values obtained from the standard.

7.2 Test procedure

Bring all **reagents and samples to room temperature** (15 °C - 30 °C) and mix well.

Mark the positions of standards/controls/samples on a protocol sheet.

Take as many microtiter strips as needed from the kit.

Store unused strips together with the desiccant bag in the closed aluminium packaging at 2 °C - 8 °C

Strips are stable until expiry date stated on the label.

For automated ELISA processors, the given protocol may need to be adjusted according to the specific features of the respective automated platform. For further details please contact your supplier or DRG.

We recommend to carry out the tests in duplicate.

1. Add each **100 µL** of the **prepared standards / controls / samples** into the respective well.
2. Cover the strips and incubate for **1 hour, shaking** on a horizontal shaker at 350 rpm with an orbit of 2 mm at room temperature (15 °C - 30 °C).
3. Discard the content of each well and wash **5 times** with **250 µL wash buffer**.
After the final washing step, remove residual wash buffer by firmly tapping the plate on absorbent paper.
4. Add **100 µL of conjugate** (diluted CONJ) into each well.
5. Cover the strips and incubate for **1 hour, shaking** on a horizontal shaker at 350 rpm with an orbit of 2 mm at room temperature (15 °C - 30 °C).
6. Discard the content of each well and wash **5 times** with **250 µL wash buffer**.
After the final washing step, remove residual wash buffer by firmly tapping the plate on absorbent paper.
7. Add **100 µL of substrate** (SUB) into each well.
8. Incubate for **10 - 20 minutes*** at room temperature (15 °C - 30 °C) in the dark.
9. Add **100 µL of stop solution** (STOP) into each well and mix well.
10. Determine **absorption immediately** with an ELISA reader at **450 nm** against 620 nm (or 690 nm) as a reference. If no reference wavelength is available, read only at 450 nm. If the extinction of the highest standard exceeds the range of the photometer, absorption must be measured immediately at 405 nm against 620 nm as a reference.

* The intensity of the colour change is temperature sensitive. We recommend observing the colour change and stopping the reaction upon good differentiation.

8 RESULTS

The following algorithms can be used alternatively to calculate the results. We recommend using the “4 parameter algorithm”.

1. *4-Parameter-algorithm*

It is recommended to use a linear ordinate for the optical density and a logarithmic abscissa for the concentration. When using a logarithmic abscissa, the zero standard must be specified with a value less than 1 (e.g. 0.01).

2. *Point-to-point-calculation*

We recommend a linear ordinate for the optical density and a linear abscissa for the concentration.

3. *Spline-algorithm*

We recommend a linear ordinate for the optical density and a linear abscissa for the concentration.

The plausibility of the duplicate values should be examined before the automatic evaluation of the results. If this option is not available with the programme used, the duplicate values should be evaluated manually.

Stool samples

The obtained results have to be multiplied by the **dilution factor of 50** to get the actual concentrations.

In case **another dilution factor** has been used, multiply the obtained result by the dilution factor used.

9 LIMITATIONS

Samples with concentrations above the measurement range (see definition below) can be further diluted and re-assayed. Please consider this greater dilution when calculating the results.

Samples with concentrations lower than the measurement range (see definition below) cannot be clearly quantified.

The upper limit of the measurement range can be calculated as:

$$\text{highest concentration of the standard curve} \times \text{sample dilution factor to be used}$$

The lower limit of the measurement range can be calculated as:

$$\text{LoB} \times \text{sample dilution factor to be used}$$

LoB see chapter “Performance Characteristics”.

10 QUALITY CONTROL

DRG recommends the use of external controls for internal quality control, if possible.

Control samples should be analysed with each run. Results, generated from the analysis of control samples, should be evaluated for acceptability using appropriate statistical methods. The results for the patient samples may not be valid if within the same assay one or more values of the quality control sample are outside the acceptable limits.

10.1 Reference range

Based on studies of stool samples of apparently healthy persons (n = 40), a median value of 61 ng/mL (\pm 46 ng/mL) was estimated.

We recommend each laboratory to establish its own reference range.

11 PERFORMANCE CHARACTERISTICS

11.1 Analytical Sensitivity

The following values have been estimated based on the concentrations of the standard without considering possibly used sample dilution factors

Limit of blank, LoB 0.118 ng/mL

Limit of detection, LoD 0.184 ng/mL

Limit of quantitation, LoQ 0.188 ng/mL

The evaluation was performed according to the CLSI-Guideline: EP17-A2. The specified accuracy goal for the LoQ was 20 % CV.

11.2 Accuracy – Precision

Repeatability (Intra-Assay); n=24

The repeatability was assessed with 3 stool samples under **constant** parameters (same operator, measurement system, day and kit lot).

Sample	Mean value [ng/mL]	CV [%]
1	117.68	6.4
2	130.91	6.0
3	38.28	3.3

Reproducibility (Inter-Assay); n=25

The reproducibility was assessed with 3 stool samples under **varying** parameters (different operators, measurement systems, days and kit lots).

Sample	Mean value [ng/mL]	CV [%]
1	74.01	13.1
2	53.30	17.6
3	61.98	18.3

11.3 Accuracy – Trueness

The trueness states the closeness of the agreement between the result of a measurement and the true value of the measurand. Therefore, 41 samples with known amounts of zonulin were measured. The recovery rate was found between 83.3 and 119.6 % for concentrations greater than the limit of quantitation.

11.4 Analytical specificity

The specificity of the antibody was tested by measuring the cross-reactivity against human haptoglobin. There was no cross-reactivity observed.

Substance tested	Concentration added	Concentration obtained	Conclusion
Human haptoglobin	2.9 mg/mL	< 0.118 ng/mL	< LoB

11.5 Linearity

The linearity states the ability of a method to provide results proportional to the concentration of analyte in the test sample within a given range. This was assessed according to CLSI guideline EP06-A with a serial dilution of 5 different stool samples.

For zonulin in stool, the method has been demonstrated to be linear from 0.37 to 4.17 ng/mL, showing a non-linear behaviour of less than $\pm 20\%$ in this interval. The table below shows the results in the linear range based on the concentrations of the standards without considering possibly used dilution factors:

Sample	Dilution	Expected [ng/mL]	Obtained [ng/mL]	Recovery [%]
A	1:50	3.00	3.00	100.00
	1:100	1.50	1.18	78.69
	1:200	0.75	0.69	92.56
	1:400	0.37	0.49	129.69
B	1:50	4.18	4.18	100.00
	1:100	2.09	1.57	75.32
	1:200	1.04	0.76	72.66
	1:400	0.52	0.54	102.90
	1:800	0.26	0.33	127.37
C	1:100	2.93	2.93	100.00
	1:200	1.46	1.49	102.05
	1:400	0.73	0.97	131.86
D	1:50	1.08	1.08	100.00
	1:100	0.54	0.39	72.31
E	1:50	2.93	2.93	100.00
	1:100	1.46	1.56	106.42
	1:200	0.73	0.90	122.49

12 PRECAUTIONS

- All reagents in the kit package are for *in vitro* diagnostic use only.
- Human materials used in kit components were tested and found to be negative for HIV, Hepatitis B and Hepatitis C. However, for safety reasons, all kit components should be treated as potentially infectious.
- Kit reagents contain sodium azide or Proclin as bactericides. Sodium azide and Proclin are toxic. Substrates for the enzymatic colour reactions are toxic and carcinogenic. Avoid contact with skin or mucous membranes.
- The stop solution consists of diluted sulphuric acid, a strong acid. Although diluted, it still must be handled with care. It can cause burns and should be handled with gloves, eye protection, and appropriate protective clothing. Any spill should be wiped up immediately with copious quantities of water. Do not breath vapour and avoid inhalation.

13 TECHNICAL HINTS

- Do not interchange different lot numbers of any kit component within the same assay. Furthermore we recommend not assembling wells of different microtiter plates for analysis.
- Control samples should be analysed with each run.
- Reagents should not be used beyond the expiration date stated on kit label.
- Substrate solution should remain colourless until use.
- To ensure accurate results, proper adhesion of plate sealers during incubation steps is necessary.
- Avoid foaming when mixing reagents.
- Do not mix plugs and caps from different reagents.
- The assay should always be performed according to the enclosed manual.

14 GENERAL NOTES ON THE TEST AND TEST PROCEDURE

- This assay was produced and distributed according to the IVD guidelines of 98/79/EC.
- The guidelines for medical laboratories should be followed.
- Incubation time, incubation temperature and pipetting volumes of the components are defined by the producer. Any variation of the test procedure, which is not coordinated with the producer, may influence the results of the test. DRG can therefore not be held responsible for any damage resulting from incorrect use.

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15 REFERENCES / LITERATURE / LITERATUR

1. Fasano, A, T Not, W Wang, S Uzzau, I Berti, A Tommasini, and S E Goldblum. 2000. "Zonulin, a Newly Discovered Modulator of Intestinal Permeability, and Its Expression in Coeliac Disease." *Lancet* **355** (9214) (April 29): 1518–9. doi:10.1016/S0140-6736(00)02169-3.
2. Wang, W, S Uzzau, S E Goldblum, and A Fasano. 2000. "Human Zonulin, a Potential Modulator of Intestinal Tight Junctions." *Journal of Cell Science* **113** Pt 24 (December): 4435–40.
3. Fasano, A. 2001. "Intestinal Zonulin: Open Sesame!" *Gut* **49** (2) (August): 159–62.
4. Freemark, Michael, and Lynne L Levitsky. 2003. "Screening for Celiac Disease in Children with Type 1 Diabetes: Two Views of the Controversy." *Diabetes Care* **26** (6) (June): 1932–9.
5. Lazzarotto, Francesca, Daniela Basso, Mario Plebani, Alessandro Moscon, Renato Zanchetta, and Corrado Betterle. 2003. "Celiac Disease and Type 1 Diabetes." *Diabetes Care* **26** (1) (January): 248–9.
6. Watts, Tammara, Irene Berti, Anna Sapone, Tania Gerarduzzi, Tarcisio Not, Ronald Zielke, and Alessio Fasano. 2005. "Role of the Intestinal Tight Junction Modulator Zonulin in the Pathogenesis of Type I Diabetes in BB Diabetic-Prone Rats." *Proceedings of the National Academy of Sciences of the United States of America* **102** (8) (February 22): 2916-21. doi:10.1073/pnas.0500178102.
7. De Magistris, Maria Teresa. 2006. "Zonula Occludens Toxin as a New Promising Adjuvant for Mucosal Vaccines." *Vaccine* **24** Suppl 2 (April 12): S2–60–1.
8. Sapone, Anna, Laura de Magistris, Michelle Pietzak, Maria G Clemente, Amit Tripathi, Francesco Cucca, Rosanna Lampis, et al. 2006. "Zonulin Upregulation Is Associated with Increased Gut Permeability in Subjects with Type 1 Diabetes and Their Relatives." *Diabetes* **55** (5) (May 1): 1443–9. doi:55/5/1443 [pii].
9. Thomas, Karen E, Anna Sapone, Alessio Fasano, and Stefanie N Vogel. 2006. "Gliadin Stimulation of Murine Macrophage Inflammatory Gene Expression and Intestinal Permeability Are MyD88-Dependent: Role of the Innate Immune Response in Celiac Disease." *Journal of Immunology* (Baltimore, Md. : 1950) **176** (4) (February 15): 2512–21.

SYMBOLS USED

Symbol	English	Deutsch	Italiano	Español	Français
	European Conformity	CE-Konformitäts-kennzeichnung	Conformità europea	Conformidad europea	Conformité normes européennes
	Consult instructions for use *	Gebrauchsanweisung beachten *	Consultare le istruzioni per l'uso	Consulte las instrucciones de uso	Consulter les instructions d'utilisation
	<i>In vitro</i> diagnostic medical device *	<i>In-vitro</i> -Diagnostikum *	Dispositivo medico-diagnostico in vitro	Producto sanitario para diagnóstico In vitro	Dispositif médical de diagnostic in vitro
	Catalogue number *	Artikelnummer *	Numero di Catalogo	Número de catálogo	Référence de catalogue
	Batch code *	Chargencode *	Codice del lotto	Codigo de lote	Numéro de lot
	Contains sufficient for <n> tests *	Ausreichend für <n> Prüfungen	Contenuto sufficiente per "n" saggi	Contenido suficiente para <n> ensayos	Contenu suffisant pour "n" tests
	Temperature limit *	Temperaturbegrenzung *	Temperatura di conservazione	Temperatura de conservación	Température de conservation
	Use-by date *	Verwendbar bis *	Utilizzare prima del	Establa hasta	Utiliser jusque
	Manufacturer *	Hersteller *	Fabbricante	Fabricante	Fabricant
	Caution *	Achtung *			
	For research use only	Nur für Forschungszwecke	Solo a scopo di ricerca	Sólo para uso en investigación	Seulement dans le cadre de recherches
<i>Distributed by</i>	Distributed by	Vertreiber	Distributore	Distribuidor	Distributeur
<i>Content</i>	Content	Inhalt	Contenuto	Contenido	Contenu
<i>Volume/No.</i>	Volume / No.	Volumen/Anzahl	Volume/Quantità	Volumen/Número	Volume/Quantité