

## Concentrated EZ Cleanser Instruction

### 【Product name】

Generic name: Concentrated EZ Cleanser for hemocyte analysis

Trade name: KT03Concentrated EZ cleanser

### 【Packing specifications】

Specification : 50mL REF:22005004

### 【Intended use】

This product can regularly remove the cell residues, blood proteins in the instrument to prevent protein precipitation in the sampling needle, pipe, counting chamber and prolong the life of the machine and assure the accuracy of the test.

### 【Inspection principle】

The surfactant can effectively remove residual blood, protein and other substances attached to the instrument so as to improve the accuracy of the measurement.

### 【Main compositions】

Protease, Lipase

### 【Storage conditions and validity】

This product should be stored between 2°C ~ 30°C, the close-vial stability is 2 years. If stored between 15°C ~ 30°C, the validity for the opened reagent is 30 days.

### 【Application range】

Applicable to 3-part KT series Auto Hematology Analyzers manufactured by Genrui Biotech Inc.

### 【Use method】

Please refer to the specific use of the instrument instruction manual

### 【Performance indicators】











This product should be colorless transparent liquid, without precipitation, granules or floccule.

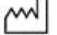

Blank value: WBC ≤ 0.2 × 10<sup>9</sup>/L, RBC ≤ 0.02 × 10<sup>12</sup>/L, HGB ≤ 1g/L, PLT ≤ 10 × 10<sup>9</sup>/L.

### 【Precautions】

- If it is frozen, please thaw it under the room temperature and shake it up slightly
- Do not use the expired reagents. Shake the antiserum reagent well before use.
- The following factors will affect the test result: Reagent expired or invalid; Pollution cause by dust in the air; sample processing unclean; mixed with other solution which are not from Shenzhen Genius Electronics Co., Ltd.
- Disposal the waste, remaining product and contaminated packaging in accordance with local regulations
- Do not directly contact reagents, if accidentally contact with eyes rinse with water immediately, if necessary, go to the hospital

### 【Symbols explanation】

Label	Meaning
	Date of manufacture
	Manufacturer
	Biological risks
<b>LOT</b>	Batch code
	Temperature limit
	Use-by date
<b>EC REP</b>	Authorized representative in the European Community
	Volume
	Consult instructions for use
	Do not re-use
	Keep away from sunlight
	Contains sufficient for <n>-tests

	Catalogue number
	CE Marking

### 【Manufacturer】



Genrui Biotech Inc.

Address: 4-10F, Building 3, Geya technology Park, Guangming District, 518106, Shenzhen, China

### 【Instruction approved and revised date】

Approved date: May 12, 2018

Revised date: NA

### 【Guarantee and Technical Support】

If invalid message repeats or need technical support, please contact Genrui Customer Service and Support Center.

