www.dialab.at

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www.dialab.at

Package insert / ENGLISH

GB DIALAB Pregnancy Test
For Self Testing

REF Z02305DL

CE 0408

contraceptives, painkillers, antibiotics or hormone therapies that do not contain hCG should not affect the test result.

- A number of medical conditions other than pregnancy, including, ovarian cyst or ectopic pregnancy (pregnancy outside the uterus) can cause elevated levels of hCG, which can result in a false positive result.

QUESTIONS & ANSWERS**1. Q: When is the test work?**

A: The DIALAB Pregnancy Test detects a hormone in your urine that your body produces during pregnancy (hCG-human Chorionic Gonadotropin). The amount of pregnancy hormone increases as pregnancy progresses.

2. Q: How soon after I suspect that I am pregnant can I take the test?

A: You can test your urine as early as the first day you miss your period. You can perform the test anytime of the day; however, if you are pregnant, first morning urine contains the most pregnancy hormone.

3. Q: Do I have to test with first morning urine?

A: Although you can test at any time of the day, your first morning urine is usually the most concentrated of the day and would have the most hCG in it.

4. Q: How accurate is the test?

A: A clinical evaluation was conducted comparing the results obtained using the DIALAB Pregnancy Test to another commercially available urine membrane hCG test. The consumer clinical trial included 140 urine specimens: both assays identified 80 positive and 60 negative results. The results demonstrated >99% overall accuracy of the DIALAB Pregnancy Test when compared to the other urine membrane hCG test.

5. Q: How sensitive is the test?

A: The DIALAB Pregnancy Test detects hCG in urine at a concentration of 25 mIU/mL or greater. The test has been standardized to WHO International Standard. The addition of LH (300 mIU/mL), FSH (1,000 mIU/mL), and TSH (1,000 mIU/mL) to negative (0 mIU/mL) hCG and positive (25 mIU/mL hCG) specimens showed no cross-reactivity.

6. Q: What should I do if the result shows that I am pregnant?

A: It means that your urine contains hCG and you are probably pregnant. See your doctor to confirm that you are pregnant and to discuss the steps you should take.

7. Q: How do I know that the test was run properly?

A: The appearance of a colored line in the control line region (C) tells you that you followed the test procedure properly and the proper amount of urine was absorbed.

8. Q: What should I do if the result shows that I am not pregnant?

A: It means that no hCG has been detected in your urine and probably you are not pregnant. If you do not start your period within a week of its due date, repeat the test with a new test strip. If you receive the same result after repeating the test and you still do not get your period, you should see your doctor.

READING THE RESULTS**PREGNANT**

Two distinct colored lines appear. One line should be in the control line region (C) and another line should be in the test line region (T). One line may be lighter than the other; they do not have to match. This means that you are probably pregnant.

NOT PREGNANT

One colored line appears in the control line region (C). No line appears in the test line region (T). This means that you are probably not pregnant.

INVALID

The result is invalid if no colored line appears in the control line region (C), even if a line appears in the test line region (T). You should repeat the test with a new test strip.

LIMITATIONS

There is the possibility that this test may produce false results. Consult your physician before taking any medical decisions.

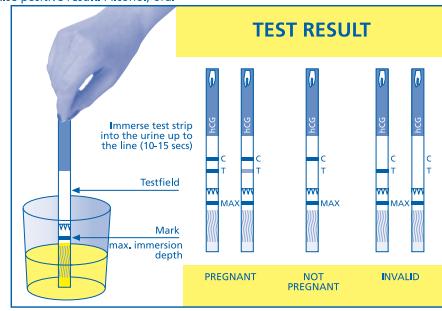
1. Drugs which contain hCG (such as Pregnyl, Profasi, Pergonal, APL) can give a false positive result. Alcohol, oral

contraceptives, painkillers, antibiotics or hormone therapies that do not contain hCG should not affect the test result.

2. Use the DIALAB Pregnancy Test to detect pregnancy when your period is late. You can begin better prenatal care as soon as you learn of your pregnancy.

3. If you get a positive result, it is advisable to visit your doctor immediately to begin your prenatal care.

3. Maintain a well-balanced diet, stop smoking, and reduce your intake of alcohol.

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