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- Jennette, J.C. and Falk, R.J. Antineutrophil Cytoplasmic Autoantibodies and Associated Diseases: a Review American Journal of Kidney Diseases 1990, Vol. XV, No. 6: 517–529.
- Gross, W.L. et al. Antineutrophil Cytoplasmic Autoantibody-Associated Diseases: A Rheumatologist`s Perspective. American Journal of Kidney Diseases 1991, Vol. XVIII, No. 2: 175–179.
- Wieslander, J. How are Antineutrophil Cytoplasmic Autoantibodies Detected? American Journal of Kidney Diseases 1991, Vol. XVIII, No. 2: 154–158.
- Lesavre, P. Antineutrophil cytoplasmic antibodies antigen specificity. American Journal of Kidney Diseases 1991, Vol. XVIII, No. 2: 159–163.
- Hagen, E.C. et al. Antineutrophil cytoplasmic autoantibodies: a review of the antigens involved, the assays, and the clinical and possible pathogenic consequences. Blood 1993, Vol.81: 1996–2000.
- Gross, W.L. et al. Immunodiagnostische and immunopathogenetische Bedeutung von Anti-Neutrophilen-Cytoplasma- Antik?rpern. Deutsche Medizinische Wochenschrift 1993, Vol. 118: 191–199.

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Instruction for use

A SOLID-PHASE ENZYME IMMUNOASSAY FOR THE QUANTITATIVE DETERMINATION OF AUTOANTIBODIES TO MYELOPEROXIDASE IN HUMAN BLOOD SERUM OR PLASMA

1. INTENDED USE

A solid-phase enzyme immunoassay for the quantitative determination of autoantibodies to myeloperoxidase in blood serum or plasma.

This kit is designed for measurement of autoantibodies to myeloperoxidase in blood serum or plasma. For possibility of use with other sample types, please, refer to Application Notes (on request). The kit contains reagents sufficient for 96 determinations and allows to analyze 41 unknown samples in duplicates.

2. SUMMARY AND EXPLANATION

Antibodies to myeloperoxidase (aMPO) are one of the two main members of anti-neutrophilic-cytoplasm antibodies (ANCA). ANCA are directed towards cytoplasmic components of neutrophilic granulocytes and monocytes. The classical method for determination of ANCA is immunofluorescence. Indirect immunofluorescence (IF) reveal two main patterns: perinuclear (p-ANCA) type and cytoplasmatic (c-ANCA) type. The main antigen for p-ANCA is myeloperoxidase (MPO) while for c-ANCA the main antigen is proteinase 3 (PR3).

A similar fluorescent p-ANCA pattern may accompany the presence of autoantibodies to other antigens –e.g., elastase, cathepsin G, lactoferrin and lysozyme. Besides, indistinguishable from p-ANCA pattern may be caused by different granulocyte specific antinuclear antibodies (GS-ANA) with atypical pattern. Therefore, an exact interpretation of IF patterns is difficult, and every positive p-ANCA pattern should be confirmed by EIA methods using purified antigens.

aMPO are mostly found in patients with eosinophilic vasculitides, including Takayasu syndrom, Churg-Strauss syndrom and periarteriitis nodosa (PAN). Combined positivity with anti-PR3 is observed in ca.70% of the patients with microscopic polyangiitis.

3. PRINCIPLE OF THE TEST

This test is based on two-site sandwich enzyme immunoassay principle. Tested specimen is placed into the microwells coated by the antigen. Antibodies from the specimen bind coated antigen on the microwell surface. Unbound material is removed by washing procedure. Second antibodies directed towards species specific Ig, labelled with peroxidase enzyme, are then added into the microwells. After subsequent washing procedure, the remaining enzymatic activity bound to the microwell surface is detected and quantified by addition of chromogen-substrate mixture, stop solution and photometry at 450 nm. Optical density in the microwell is directly related to the quantity of the measured analyte in the specimen.

4. WARNINGS AND PRECAUTIONS

4.1. For professional use only.

4.2. This kit is intended for in vitro diagnostic use only.

4.3. INFECTION HAZARD: There is no available test methods that can absolutely assure that Hepatitis B and C viruses, HIV-1/2, or other infectious agents are not present in the reagents of this kit. All human products, including patient samples, should be considered potentially infectious. Handling and disposal should be in accordance with the procedures defined by an appropriate national biohazard safety guidelines or regulations.

4.4. Avoid contact with stop solution containing 5,0% H₂SO₄. It may cause skin irritation and burns.

4.5. Wear disposable latex gloves when handling specimens and reagents. Microbial contamination of reagents may give false results.

4.6. Do not use the kit beyond the expiration date.

4.7. All indicated volumes have to be performed according to the protocol. Optimal test results are only obtained when using calibrated pipettes and microplate readers.

4.8. Do not smoke, eat, drink or apply cosmetics in areas where specimens or kit reagents are handled.

4.9. Chemicals and prepared or used reagents have to be treated as hazardous waste according to the national biohazard safety guidelines or regulations.

4.10. Do not mix reagents from different lots.

4.11. Replace caps on reagents immediately. Do not swap caps.

4.12. Do not pipette reagents by mouth.

4.13. Specimens must not contain any AZIDE compounds – they inhibit activity of peroxidase.

4.14. Safety Data Sheet for this product is available upon request directly from XEMA Co., Ltd.

4.15. The Safety Data Sheet fit the requirements of EU Guideline 91/155 EC.

5. KIT COMPONENTS 5.1. Contents of the Kit

Symbol	Description	Qty	Units	Colour code	Stability of opened/diluted components
1	SORB MTP aMPO EIA strips, 8x12 wells	1	pcs		until exp.date
2	CAL 1-6 Calibrator set, 1.1 ml each. The set contains 6 calibrators: 0; 6.25; 12.5; 25; 50; 100 U/ml	6	pcs	red (C1 – colourless)	2 months
3	CONTROL Control serum (1.1 ml)	1	pcs	colourless	2 months
4	CONJ HRP Conjugate, 11 ml	1	pcs	red	until exp.date
5	DIL SPE EIA buffer 50 ml	1	pcs	blue	until exp.date
6	SUBS TMB Substrate solution, 11 ml	1	pcs	colourless	until exp.date
7	BUF WASH 21X Washing solution concentrate 21x, 22 ml	1	pcs	colourless	Concentrate – until exp.date Diluted washing solution – 1 month at +2, +8°C or 5 days at RT
8	STOP Stop solution, 11 ml	1	pcs	colourless	until exp.date
9	N003 Plate sealing tape	2	pcs		N/A
10	K1331 Instruction aMPO EIA	1	pcs		N/A
11	K133Q QC data sheet aMPO EIA	1	pcs		N/A

5.2. Equipment and material required but not provided

- Distilled or deionized water;
- Automatic or semiautomatic multichannel micropipettes, 100–250 µl, is useful but not essential;
- Calibrated micropipettes with variable volume, range volume 25–250 µl;
- Calibrated microplate photometer with 450 nm wavelength and OD measuring range 0–3.0.

5.3. Storage and stability of the Kit

Store the whole kit at +2...+8 °C upon receipt until the expiration date.

After opening the pouch keep unused microtiter wells **TIGHTLY SEALED BY ADHESIVE TAPE (INCLUDED)** to minimize exposure to moisture.

6. SPECIMEN COLLECTION AND STORAGE

This kit is intended for use with serum or plasma (ACD- or heparinized). Grossly hemolytic, lipemic, or turbid samples should be avoided.

Specimens may be stored for up to 48 hours at +2...+8 °C before testing. For a longer storage, the specimens should be frozen at -20 °C or lower. Repeated freezing/thawing should be avoided.

7. TEST PROCEDURE**7.1.** Reagent Preparation

- All reagents (including unsealed microstrips) should be allowed to reach room temperature (+18...+25 °C) before use.
- All reagents should be mixed by gentle inversion or vortexing prior to use. Avoid foam formation.
- It is recommended to spin down shortly the tubes with calibrators on low speed centrifuge.
- Prepare washing solution from the concentrate BUF WASH 21X by 21 dilutions in distilled water.

7.2. Procedural Note:

It is recommended that pipetting of all calibrators and samples should be completed within 3 minutes.

7.3. Assay flowchart

See the example of calibration graphic in Quality Control data sheet.

7.4. Assay procedure

1	Put the desired number of microstrips into the frame; allocate 14 wells for the calibrators CAL 1-6 and control samples CONTROL and two wells for each unknown sample. DO NOT REMOVE ADHESIVE SEALING TAPE FROM UNUSED STRIPS.
2	Dilute all samples using buffer DIL SPE (EIA buffer) 101 fold. (5 µl of sample + 500 µl of diluent) . Do not dilute control sample and calibrators.
3	If suggested analyte concentration in the sample exceeds the highest calibrator, additionally dilute this sample accordingly, using DIL SPE (EIA buffer). Use of other buffers or reagents for sample dilution may lead to incorrect measurement.
4	Pipet 100 µl of calibrators CAL 1-6, control sample CONTROL and unknown samples into the wells. Cover the wells by plate adhesive tape (included into the kit).
5	Incubate 30 minutes at +18...+25 °C.
6	Prepare washing solution by 21x dilution of washing solution concentrate BUF WASH 21X by distilled water. Minimal quantity of washing solution should be 250 µl per well. Wash strips 3 times
7	Dispense 100 µl of CONJ HRP into the wells. Cover the wells by plate adhesive tape.
8	Incubate 30 minutes at +18...+25 °C.
9	Wash the strips 5 times.
10	Dispense 100 µl of SUBS TMB into the wells
11	Incubate 10-20 minutes at +18...+25 °C
12	Dispense 100 µl of STOP into the wells.
13	Measure OD (optical density) at 450 nm.
14	Set photometer blank on first calibrator
15	Apply point-by-point method for data reduction.

7.5. Sample processing

Material type	Notes on material collection, storage and handling	Sample dilution example	Sample into the well, µl	Calculation factor
blood serum or plasma	Grossly hemolytic, lipemic, or turbid samples should be avoided and should be treated by centrifugation before testing.	5 µl of sample + 500 µl of diluent	100	1

8. QUALITY CONTROL

It is recommended to use control samples according to state and federal regulations. The use of control samples is advised to assure the day to day validity of results.

The test must be performed exactly as per the manufacturer's instructions for use. Moreover the user must strictly adhere to the rules of GLP (Good Laboratory Practice) or other applicable federal, state, and local standards and/or laws. This is especially relevant for the use of control reagents. It is important to always include, within the test procedure, a sufficient number of controls for validating the accuracy and precision of the test.

The test results are valid only if all controls are within the specified ranges and if all other test parameters are also within the given assay specifications.

9. CALCULATION OF RESULTS


9.1. Calculate the mean absorbance values (OD450) for each pair of calibrators and samples.

9.2. Plot a calibration curve on graph paper: OD versus autoantibodies to myeloperoxidase concentration.

9.3. Determine the corresponding concentration of autoantibodies to myeloperoxidase in unknown samples from the calibration curve. Manual or computerized data reduction is applicable on this stage. Point-by-point or linear data reduction is recommended due to non-linear shape of curve.

9.4. Below is presented a typical example of a standard curve with the XEMA Co. Not for calculations!

Calibrators	Value	Absorbance Units (450 nm)
CAL 1	0 U/ml	0.10
CAL 2	6.25 U/ml	0.17
CAL 3	12.5 U/ml	0.25
CAL 4	25 U/ml	0.45
CAL 5	50 U/ml	1.08
CAL 6	100 U/ml	2.16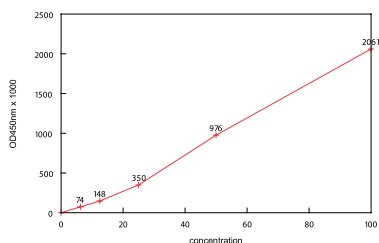

10. EXPECTED VALUES

Therapeutical consequences should not be based on results of IVD methods alone – all available clinical and laboratory findings should be used by a physician to elaborate therapeutically measures. Each laboratory should establish its own normal range for aMPO. Based on data obtained by XEMA, the following normal range is recommended (see below).

Sex, age	Units, U/ml	
	Lower limit	Upper limit
Healthy donors	-	10.0

11. PERFORMANCE CHARACTERISTICS

11.1. Analytical sensitivity

Sensitivity of the assay was assessed as being 3 IU/ml.

11.2. Linearity

Linearity was checked by assaying dilution series of 5 samples with different autoantibodies to myeloperoxidase concentrations. Linearity percentages obtained ranged within 90 to 110%.

11.3. Recovery

Recovery was estimated by assaying 5 mixed samples with known autoantibodies to myeloperoxidase concentrations. The recovery percentages ranged from 90 to 110%.

12. LITERATURE

1. Jennette, J.C. and Falk, R.J. Antineutrophil Cytoplasmic Autoantibodies and Associated Diseases: a Review American Journal of Kidney Diseases 1990, Vol. XV, No. 6: 517–529.
2. Gross, W.L. et al. Antineutrophil Cytoplasmic Autoantibody-Associated Diseases: A Rheumatologist's Perspective. American Journal of Kidney Diseases 1991, Vol. XVIII, No. 2: 175–179.
3. Wieslander, J. How are Antineutrophil Cytoplasmic Autoantibodies Detected? American Journal of Kidney Diseases 1991, Vol. XVIII, No. 2: 154–158.
4. Lesavre, P. Antineutrophil cytoplasmic antibodies antigen specificity. American Journal of Kidney Diseases 1991, Vol. XVIII, No. 2: 159–163.
5. Hagen, E.C. et al. Antineutrophil cytoplasmic autoantibodies: a review of the antigens involved, the assays, and the clinical and possible pathogenic consequences. Blood 1993, Vol.81: 1996–2000.
6. Gross, W.L. et al. Immunodiagnostische and immunopathogenetische Bedeutung von Anti-Neutrophilen-Cytoplasma- Antikörpern. Deutsche Medizinische Wochenschrift 1993, Vol. 118: 191–199.